**NOTE:** Please read the following instructions carefully prior to assembly.

**This item should be fitted by an experienced clinician or technician. This document contains important information that must be passed on to the user at delivery.**

Fitting the Flo-tech PT Pelvic Belt

Position the strap below the Anterior Superior Iliac Spine (ASIS) and attach to the wheelchair at the desired angle. For best results, it is recommended that the strap is fitted at an angle of 45-60 degrees to the seat.

Fitting the pelvic belt across the upper thighs in front of the greater trochanters may help to prevent sliding.

**TRANSPORTATION**

The Invacare pelvic positioning belts are suitable for use in transport only when fitted to a secure point on the wheelchair frame. A separate passenger restraint should always be used, please refer to wheelchair user manual – transportation section.

**MAINTENANCE**

Before each use:

Undertake a visual check for torn fabric, loose or worn screws.

Bi-Annually:

Ensure that pelvic belts are inspected for loose or worn fastenings and that the torque settings are checked by a suitably qualified and competent person. For heavier and high impact users, the period between inspections should be reduced accordingly.

The Expected Product Life-Cycle for this product is 30 months.

**LIMITATIONS OF USE**

The Invacare pelvic positioning belts are suitable for use in transport only when fitted to a secure point on the wheelchair frame. Please refer to section 2 above “Transportation”.

**TRAINING**

If for any reason you should require additional training regarding the assembly of this hardware, please contact us at the number listed below, for free advice and support.

**COMPATIBILITY**

Invacare pelvic belts are compatible with wheelchairs that have round frame tubing of 19mm, 22mm or 25mm diameter.

**QUALITY**

In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice. All rights reserved.

### CLEANING AND CARE

The Belt range of products may be cleaned with a neutral detergent (e.g. soapy hot water) and / or a single use wipe. All Invacare belts with Plastic Squeeze Buckle may be machine washed at 60°C and should be air dried. Not suitable for tumble drying. Refer to washing label.

**NOTE: Belts with Auto Style Release should be hand washed with care and air dried.** The mounting hardware may be cleaned with common household cleaners and a soft cloth. Should the product become contaminated, please ask your infection control officer for advice. Store in a dry environment.

### CAUTION

- Fitting a pelvic belt over the lower abdomen may restrict the diaphragm and cause breathing difficulties.

### SAFETY

- In the interest of safety, it is recommended that the pelvic belt is adjusted whenever it is used. The pelvic belt should be tightened sufficiently to prevent the user from sliding down between the belt and the seat.
- The pelvic belt provided with your wheelchair is for postural support and is not designed as an occupant restraint for use in transportation. If the user remains in their wheelchair whilst in transportation, a separate, suitable passenger restraint must be used.
- This item must not be used as a restraint.

A

Auto Style  
Release

B

Plastic Squeeze  
Buckle

C

Twin Lock  
Release

D

Mounting Clamps  
Qty: 2

## INSTRUCTIONS FOR ASSEMBLY

STEP 1—Remove the pelvic belt and components from the packaging.

STEP 2— Should this product reach you in an unsatisfactory condition, please contact the supplier.

STEP 3—Attach the tri slot mounting plates (see fig. 1) into the desired position onto the wheelchair frame, by means of the Universal Mounting Clamps, which are suitable for round tube frames of 19mm, 22mm, and 25mm (see fig. 2)

**It is recommended that all screws can be tightened to 8.5Nm.**

Figure 1

Figure 2

**INSTRUCTIONS FOR USE**

**Adjustment**

1. Ensure that the user is sitting correctly, with their bottom as far back in the seat as possible.
2. Position the lap belt around the user in the desired position and push the two sides of the buckle together. Pull to confirm buckle locked.
3. Adjust the length of the lap strap using the pulls on the side/s of the buckle (see fig. 3). The belt should fit without causing discomfort. There should be sufficient room to slide two fingers between the user and the belt. For ease, the buckle should be kept in a central position.

**Figure 3**

**To Release the Belt**

See table below for release instructions.

<b>Pelvic Belt Style</b>	<b>Release Instructions</b>
Squeeze Buckle	Using the thumb and forefinger, squeeze the two plastic “buttons” at the top and bottom of the buckle.
Auto Style Release	Push the button in the centre of the buckle.
Twin Lock Plastic Buckle	Using the thumb and forefinger, squeeze the two plastic “buttons” at the top and bottom of the buckle whilst simultaneously pressing the centre button.

**WARRANTY**

Flo-tech PT Pelvic Belts are covered by a warranty period of 12 months.

Invacare Ltd  
Penoced Technology Park  
Pencoed, Bridgend CF35 5HZ, UK  
Tel: +44 (0) 1656 776222 Fax: +44 (0) 1656 776220  
E-mail: uk@invacare.com Web: www.invacare.com

This product is manufactured to comply with the Medical Device Directive

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