**P225M – P226M**  
**P225L - P226L**  
**P240**  
**P227S - P228B**  
**P228G - P228R**  
**P2800 - P2802**  
**P2803 - P2804 - P2805**  
**P2831 - P2832**  
**P252 - P253 - P254**  
**P255 - P257 - P270**  
**P256 - P260**  
**P264A - P265A**  
**P328 - P328J**  
**P306**

<b>F</b>	1.	Guide utilisateur
<b>GB</b>	2.	User's manual
<b>D</b>	3.	Gebrauchsanweisung
<b>E</b>	4.	Manual del usuario
<b>I</b>	5.	Manuale dell'utente
<b>NL</b>	6.	Gebruiksaanwijzing
<b>P</b>	7.	Guia do utilizador
<b>S</b>	8.	Bruksanvisning
<b>DK</b>	9.	Brugsanvisning
<b>FIN</b>	10.	Käyttäjän käsikirja
<b>N</b>	11.	Bruksanvisning

## I. FRANCAIS

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## 2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

**The following operating and maintenance instructions must be observed:**

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load  or the maximum load  shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

### OPENING THE FRAME (EXCEPT P435M)

Pull the walking frame's two side uprights apart until they lock. The P402 has two positions; to open it wider, press down the central plastic ring and move the uprights apart until they lock in the new position.

### SETTING THE HEIGHT OF THE LEGS

Press the spring button until the leg releases, adjust the leg to the desired height and check that the spring button is positioned correctly. All the legs should be set to the same height.

### USE

Before using the walking frame, check that it is locked in the open position by gripping the sides and trying to close them.

The walking frame should only be used on hard, flat surfaces. Do not use it on stairs.

### FITTING ACCESSORIES

#### Fixed front wheels on the P435M frame (O435CE)

- Replace the two front legs with the two-wheeled extension pieces, with the wheels pointing outwards.
- Adjust the height; the frame's handles should be horizontal.

#### Front wheels for frames 6291E-A and 6291E-JR (6270E: fixed front wheels / 6267E: swivelling front wheels)

- Replace the two front legs with the two-wheeled extension pieces, with the wheels pointing outwards, following the following convention: RH = right-hand wheel and LH = left-hand wheel (as viewed from the back of the frame). Adjust the height; the frame's handles should be horizontal.

• Then replace the standard end caps on the rear legs with the low-friction caps supplied. Warning: the low-friction caps are designed to slide when you are walking and do not have the same grip as the standard caps. Take particular care when using them.

#### Rear wheels with pressure brakes for frames 6291E-A and 6291E-JR (6265E)

NB: 6265E braked rear wheels can only be fitted to frames with 6267E front wheels.

- Replace the rear legs with the wheeled extension pieces. Adjust the height. The frame's handles should be horizontal.

#### Shopping bag for frames 6291E-A and 6291E-JR (6013)

Attach the 6013 shopping bag to the front and sides of the frame with Velcro.

#### Front armrests for frames 6291E-A and 6291E-JR (6027)

- Slide the two aluminium armrests onto the vertical bars on each side of the frame, pulling them apart gently until they are just above the horizontal plastic bar. Turn the armrests inwards so that they face each other. Insert the two cowls on each end of the 31cm tube. Fit the 31cm tube into the armrests and fix it in place using 2 screws and nuts.
- Slide the armrest support into the 75cm tube and tighten it. Slide the tube with the handle under the support: tighten as soon as you can see the lug.
- Slide the 75cm tube into the flat armrests and refit onto the support.
- Fit the 75cm tube vertically. Fit a "U" around the 31cm tube into the two holes in the 75cm tube. Tighten temporarily with 2 nuts.
- Fit a "U" around the plastic handle on the upper part of the frame into the 2 holes in the armrest. Tighten temporarily with 2 nuts.
- Position everything correctly and tighten.

### FOLDING

#### P402 and P408

Hold down the central plastic ring and fold the side uprights inwards.

#### 6291E-A and 6291E-JR

Press down the top of one side and fold the side inwards. Repeat for the other side.

### MAINTENANCE

Use a soft cloth to clean away any dirt or dust. Use soap and water to remove stubborn stains. Dry with a soft cloth. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

### USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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