H720T2 H720T4 H720T4C

- | | | |
|------------|-----|----------------------|
| F | 1. | Guide utilisateur |
| GB | 2. | User's manual |
| D | 3. | Gebrauchsanweisung |
| E | 4. | Manual del usuario |
| I | 5. | Manuale dell' utente |
| NL | 6. | Gebruiksaanwijzing |
| P | 7. | Guia do utilizador |
| S | 8. | Bruksanvisning |
| DK | 9. | Brugsanvisning |
| FIN | 10. | Käyttäjän käsikirja |
| N | 11. | Bruksanvisning |

I. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load  or the maximum load  shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

ASSEMBLY

- Slide the back over the tubes behind the seat.
- Tighten the two star knobs firmly to fix the back in place. Warning: the back must be correctly fitted over the rear tubes to ensure that the armrests are held correctly.
- Raise the armrests and lock them in place. They should fit tightly, with the two spring buttons visible. If this is not the case, check that the back is fitted correctly.
- Place the footrests on the lugs provided at the front of the commode; they should lock automatically.
- Clip the perforated seat onto the frame. Ensure that it is correctly fitted before use.
- Fit the removable seat over the perforated seat and slide the toilet bowl in place.

USE

- The armrests can be folded away. To unlock them, pull the black plastic knob on the front of the armrest.
- The footrests can also be folded towards the back of the commode; unlock the retention system on the side of the commode.
- The footrests are height adjustable and can be folded away vertically.

Locking / transport

- The commode has two-rear brakes (H720T2) or four brakes all around (H720T4 and H720T4C). At least two brakes are required to lock the commode in place. Warning: use in damp conditions can affect the efficiency of the brakes.
- Before transportation, ensure that at least 2 brakes are on, that the wheels are facing forwards and that the footrests are raised.
- Never support yourself on the footrests while sitting down or getting up.

Moving

- Do not push the commode down slopes or over rough surfaces. Do not attempt to mount obstacles over 5mm in height.
- Check that no part of the body is outside the commode when passing through doorways (especially near the armrests).
- The commodes are designed for indoor use only and should not be used as shower seats.

MAINTENANCE

Clean regularly with a normal household cleaner. The toilet bowl can be put through an autoclave (1 hour at 80°). For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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