Aquatec® Ocean E-VIP

| | | |
|----|---|-----|
| en | Mobile Shower and Toilet Chair User Manual | 3 |
| de | Mobiler Dusch- und Toilettenstuhl Gebrauchsanweisung..... | 41 |
| fr | Chaise roulante pour douche et toilettes Manuel d'utilisation..... | 85 |
| it | Sedia da doccia e toilette mobile Manuale d'uso..... | 127 |
| es | Silla de ducha e inodoro móvil Manual del usuario | 169 |
| nl | Mobiele douche- en toiletstoel Gebruiksaanwijzing | 211 |

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved for
future reference.

AQUATEC®

Yes, you can.®

©2022 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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Contents

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

| | |
|---|-----------|
| 1 General | 4 |
| 1.1 Introduction | 4 |
| 1.2 Symbols in this Document | 4 |
| 1.3 Warranty Information | 4 |
| 1.4 Compliance | 5 |
| 1.4.1 Product-Specific Standards | 5 |
| 1.5 Service Life | 5 |
| 1.6 Limitation of Liability | 5 |
| 2 Safety | 6 |
| 2.1 Safety Information | 6 |
| 2.2 Safety Information on Electromagnetic Compatibility | 7 |
| 2.3 Labels and Symbols on Product | 8 |
| 3 Components | 10 |
| 3.1 Product Description | 10 |
| 3.2 Intended Use | 10 |
| 3.3 Components | 10 |
| 4 Setup | 12 |
| 4.1 Safety Information | 12 |
| 4.2 Assembling the Product | 12 |
| 4.3 Assembling Seat Frame | 12 |
| 4.4 Installing the Headrest | 13 |
| 4.5 Fitting Chest Belt | 14 |
| 4.6 Fitting Seat Plate | 15 |
| 4.7 Adjusting Backrest Cover | 16 |
| 4.8 Setting Armrest Height | 17 |
| 4.9 Fitting Footrests | 18 |
| 4.10 Fitting Electronics Cover | 19 |

| | |
|---|-----------|
| 4.11 Removing Battery Pack | 20 |
| 4.12 Charging Battery | 21 |
| 5 Usage | 23 |
| 5.1 Safety Information | 23 |
| 5.2 Turning Armrests | 24 |
| 5.3 Turning Footrests | 24 |
| 5.4 Using Hand Control | 24 |
| 5.5 Engaging/Disengaging the Castor Locks | 26 |
| 6 Maintenance | 27 |
| 6.1 Servicing and Inspection | 27 |
| 6.2 Cleaning and Disinfection | 27 |
| 6.2.1 General safety information | 27 |
| 6.2.2 Cleaning Intervals | 28 |
| 6.2.3 Cleaning Instructions | 28 |
| 6.2.4 Disinfection Instructions | 29 |
| 7 After Use | 30 |
| 7.1 Storage and Transport | 30 |
| 7.2 Reconditioning | 30 |
| 7.3 Disposal | 30 |
| 8 Troubleshooting | 31 |
| 8.1 Identifying and Repairing Malfunctions | 31 |
| 9 Technical Data | 34 |
| 9.1 Dimensions and Weights | 34 |
| 9.2 Environmental Parameters | 34 |
| 9.3 Electronic Data | 35 |
| 9.4 Materials | 35 |
| 10 Electromagnetic compatibility (EMC) | 36 |
| 10.1 General EMC information | 36 |
| 10.2 Electromagnetic emission | 36 |
| 10.3 Electromagnetic Immunity | 37 |

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

1.3 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.4 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

1.4.1 Product-Specific Standards

The product meets the requirement of the standard DIN EN 12182 (Assistive products for persons with disability) and related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

1.5 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Injury

- Do not use if defective.
- In the event of a malfunction, contact your provider immediately.
- Do not make any unauthorized changes or alterations to the product.
- Only use accessories intended for this product.

WARNING!

Risk of Injury

- Do not stand on the product.
- Do not use as a climbing aid.
- Do not use outdoors.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

**IMPORTANT!**

When combining this product with another product, the restrictions of both products apply for the combination. E.g. the maximum user weight of one product in the combination may be lower.

- Only use combinations allowed by Invacare. Contact your Invacare distributor for information.
- Before use, read the user manual of each product and check the restrictions. The full compliance with the information, instructions and recommendations contained in both user manuals is necessary for a safe use.

2.2 Safety Information on Electromagnetic Compatibility

**WARNING!****Risk of Malfunction due to Electromagnetic Interference**

Electromagnetic interference between this product and other electrical equipment can occur and disturb the electrical adjustment functions of this product. To prevent, reduce or eliminate such electromagnetic interference:

- Only use original cables, accessories and spare parts, to not increase electromagnetic emission or reduce electromagnetic immunity of this product.
- Do not use portable radio frequency (RF) communications equipment closer than 30 cm to any part of this product (including cables).
- Do not use this product near active high-frequency surgical equipment and the RF shielded room of a system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.
- If disturbances occur, increase the distance between this product and the other equipment or switch it off.

WARNING!

Risk of Malfunction

Electromagnetic interference may cause improper operation.

- Do not use this product adjacent to or stacked with other electrical equipment. If such use is necessary this product and the other equipment must be closely observed to verify that they are operating normally.

We are not aware of any risk when using products like a cardiac pacemaker.

For more information about electromagnetic compatibility, refer to *10 Electromagnetic compatibility (EMC), page 36.*

2.3 Labels and Symbols on Product

Identification Label

The identification label contains important information:

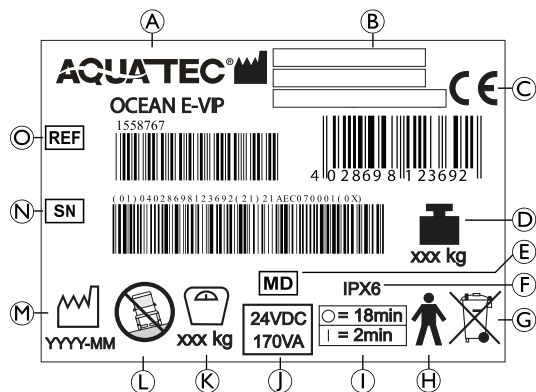
Fig. 2-1

| | |
|---|--|
| Ⓐ | Product designation |
| Ⓑ | Manufacturer and Manufacturer address |
| Ⓒ | European Conformity |
| Ⓓ | Total weight of the product with the maximum safe working load applied |
| Ⓔ | Medical Device |
| Ⓕ | Type of protection |
| Ⓖ | Note (disposal) |
| Ⓗ | Type B applied part |
| Ⓘ | Interval of use (continuous operation of the engine is not allowed): max. 2 minutes continuous use followed by a 18 minute break |

| | |
|---|--|
| Ⓛ | Nominal voltage / rated power, refer to <i>9 Technical Data, page 34</i> |
| Ⓚ | Maximum user weight |
| Ⓛ | Note (only suitable for use on flat surfaces) |
| Ⓜ | Date of manufacture |
| Ⓝ | Serial number |
| Ⓞ | Reference number |

The identification label is applied to the rear left side tube.

Symbols on Product

| | |
|--|------------------|
|  | Read user manual |
|--|------------------|

3 Components

3.1 Product Description

Aquatec Ocean E-VIP is a mobile shower- and toilet chair with electric tilt and seat height adjustment functions, stainless steel frame, tension adjustable backrest, flip-up armrests, foldable textured footplates, designed for people with disabilities that restrict showering an/or toilet use.

3.2 Intended Use

Aquatec Ocean E-VIP shower- and toilet chair is intended to be used as an aid for showering, going to the toilet and for all personal hygiene activities with assistance. The chair must only be used indoors.

Intended Users

Users are adolescents or adults with permanent or transitional physical mobility limitations or restrictions.

Adolescents or adults are persons having a physical size equal to or more than 146 cm, a weight equal to or more than 40 kg and a body mass index (BMI) equal to or more than 16.

Indications

- Restricted or limited mobility (e.g. legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

Contraindications

There are no contraindications known when using the product as intended.

The maximum user weight is 150 kg.

3.3 Components

The following components are included within the scope of delivery:

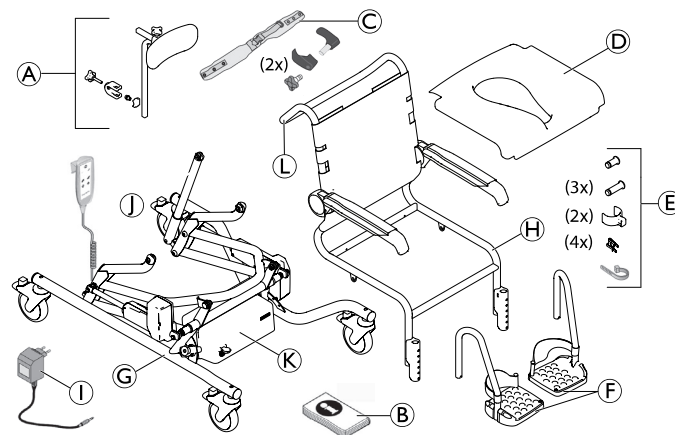
Fig. 3-1

| | |
|-----|--|
| (A) | Headrest |
| (B) | User manual |
| (C) | Chest belt with fixation (2x) |
| (D) | Seat plate |
| (E) | Bolt (4 x), safety clips (4 x), clips for footrests (2 x), cable tie (1 x) packed in bag |

| | |
|--|---|
| Ⓕ | Footrest with heel strap (2 x) |
| Ⓖ | Chassis with hand control and control box with battery packs + drives |
| Ⓗ | Seat frame with backrest cover and armrests |
| Ⓘ | Charger |
| Ⓢ | Tilt drive |
| Ⓚ | Electronics cover |
| Ⓛ | Push bar |
| Applied parts: Headrest, backrest cover, armrest, seat plate, footrest | |

 Due to regional differences, refer to your local Invacare website or catalog for details about available accessories or contact your Invacare provider.

4 Setup

4.1 Safety Information

WARNING!

- Check the parts for transport damage before commissioning and contact your provider if required.
- Ensure during assembly that the parts are positioned correctly with respect to one another.

CAUTION!

Risk of Pinching

- Take care during assembly.

4.2 Assembling the Product

This section describes the assembly. Disassembly takes place in the reverse order.

The armrest height, footrest height, footrest angle, heel strap, tension of backrest cover and accessories must be adjusted to optimally fit the user.

4.3 Assembling Seat Frame

WARNING!

Risk of Injury or Damage

- Bolts slipping out or coming loose can cause injury or damage.
- Make sure that the safety clips engage and are correctly fitted (see graphic below).

1. Press red foot lever down on the castors to secure all castors.
2. Cut cable ties which are used as transport lock.
- 3.

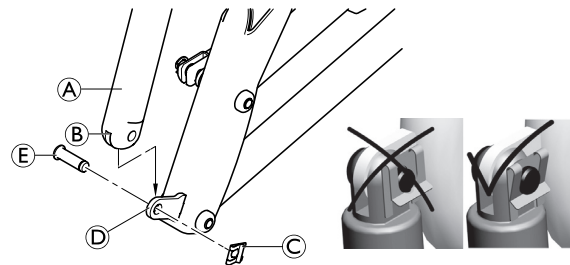
Fig. 4-1

Remove the drive for the seat tilt **A** from the chassis.

4. Push fork **B** at the lower end of drive **A** onto lip **D** at the back end of the chassis and align the holes.
5. Insert bolt **E** (35 mm) into holes from the outside and push it in completely.
6. Attach a safety clip **C** on the inside.
- 7.

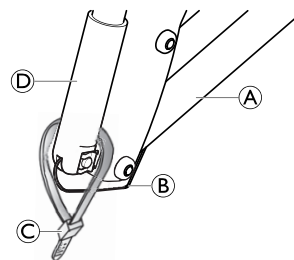
Fig. 4-2

Insert cable **B** of drive **D** as far as possible into chassis **A**.

8. Use provided cable tie **©** to secure the cable of the drive in this position.

i The armrests face forward when folded down.

9.


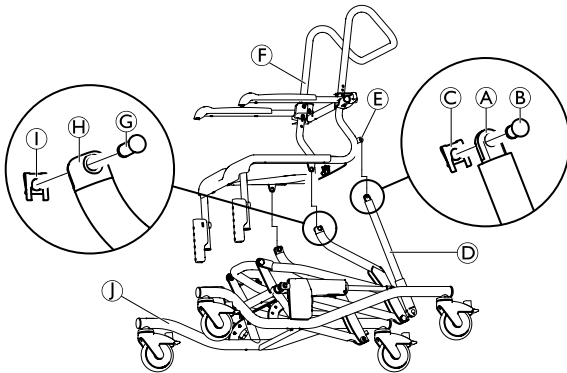
Fig. 4-3

Insert both sides of seat frame **Ⓕ** into receivers **Ⓗ** on chassis **Ⓙ**.

10. Insert two 35 mm bolts **Ⓒ** into receivers **Ⓗ** from the outside. Push in completely and attach safety clips **Ⓛ** on the inside.
11. Tilt the seat frame until the holes in bracket **Ⓔ** on the seat frame and the holes in fork **Ⓐ** on seat tilt drive **Ⓓ** are aligned.
12. Insert 23 mm bolt **Ⓑ** into the holes from the outside and attach safety clip **Ⓒ** on the inside.

4.4 Installing the Headrest

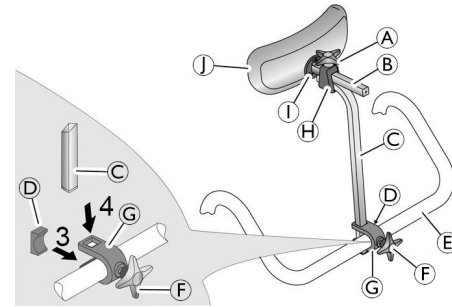1. Use the star handle **Ⓕ** to insert the screw a little into the bracket **Ⓖ**.
2. Push the bracket with the screw onto the handle of the backrest **Ⓔ** from behind with the screw engaging in the hole of the backrest's handle.
3. Insert the plate **Ⓓ** into the bracket from the front so that both lugs of the plate face down and up and the curved plate side touches the handle of the backrest.
4. Push the height adjustment tube **Ⓒ** from above into the recesses of the bracket.
5. Use the star handle to tighten the screw until the height adjustment tube is fixed in place.
6. Push the pad **Ⓙ** and the pad holder **Ⓟ** into the holder **Ⓗ** on the height adjustment tube.
7. Use the star handle **Ⓐ** to tighten the upper screw until the pad holder is fixed in place.

 To adapt it to the user, the height adjustment tube can be fitted with the curve facing the front or the back.

The headrest pad ① can be removed and replaced by other pad shapes (optional). To remove it, loosen the retaining bolts ①.

Adjusting the Length of the Headrest

 The headrest tubes ③ and ④ can be shortened if required (refer to the service manual). Shortening the headrest tube must be performed by a qualified technician.

4.5 Fitting Chest Belt

Risk of Damage

- Install both mounting brackets at the same height
- Installation of the left and right mounting brackets is laterally inverted.

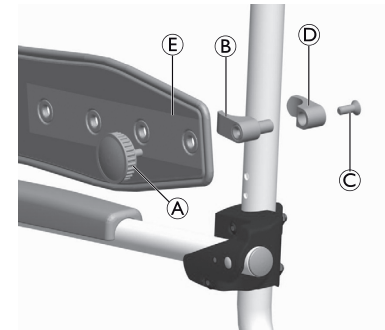
Fig. 4-4

1. Open the belt buckle on the backrest cover at the height where the chest belt is to be fitted.
2. Pass this belt on the inside of the backrest frame and close it again.
3. Place inner part ④ of the mounting bracket around the backrest frame so that the hole faces backwards.
4. Place the outer part ③ of the mounting bracket in the hole of the inner part.
5. Attach the two parts of the mounting bracket in the required position on backrest frame with screw ⑤.
6. Attach chest belt ⑥ with hand knob ⑦ to the mounting bracket.

Adjusting Chest Belt Position

Risk of Damage

- Adjust the position of the chest belt to suit the user.
- Adjust the position of the two mounting brackets equally on both sides.
- The position of the right and left mounting brackets is adjusted in the same way.

1. Open the belt buckle on the backrest cover which is passed on the inside of the backrest frame at the height where the chest belt is secured.
2. Open the belt buckle on the backrest cover at the height where the chest belt is to be fitted.
3. Pass this belt on the inside of the backrest frame and close it again.
4. Remove hand knob **A**.
5. Remove chest belt **E**.
6. Remove both parts of the mounting bracket **B** and **D** and move them to required position.
7. Place inner part **D** of the mounting bracket around the backrest frame so that the hole faces backwards.
8. Place the outer part **B** of the mounting bracket in the hole of the inner part.
9. Attach the two parts of the mounting bracket in the required position on backrest frame with screw **C**.
10. Attach chest belt **E** with hand knob **A** to the mounting bracket.
11. Pass the open belt of the backrest cover on the outside of the backrest frame and close it again.
12. Tighten all belts.

Adjusting Chest Belt Length

Adjust the length of the chest belt to suit the user.

The length of the chest belt is adjusted via the belt buckle in the chest belt. If the belt buckle adjustment range is insufficient, adjust the length at the mounting brackets as follows:

1. Remove hand knob **A**.
2. Use holes in chest belt **E** to select the length.
3. Use hand knob to fix chest belt to mounting bracket.

4.6 Fitting Seat Plate

CAUTION!

Risk of Pinching Fingers

- Do not put fingers between the seat surface and seat frame when pressing down the seat plate.

Installation

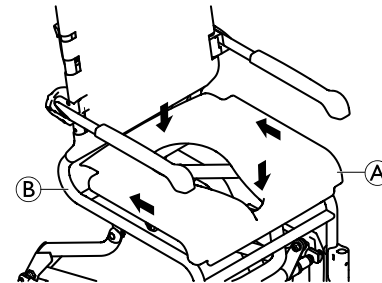
Fig. 4-5

1. Place seat plate (A) on seat frame (B) with the cut out facing to the front.
2. Slide the seat plate back as far as it goes.
3. Using the flat of your hand, press down on the sides and then the front and back of the seat plate until the seat plate clicks into the seat frame.

Removal

1. Pull up the seat plate on both sides.

4.7 Adjusting Backrest Cover

The tension of the backrest can be adjusted as desired.

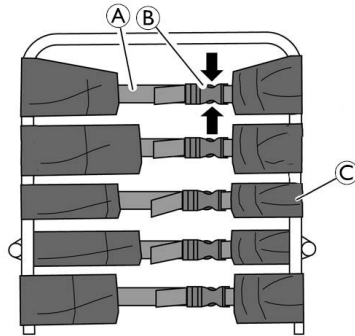
Fig. 4-6

WARNING!

- Make sure that at least three belts are always passed by the backrest frame (on the outside) and their belt buckles are closed.

To attach accessories, the third belt from the top can be opened and the backrest cover can be passed by the backrest frame on the inside at point (C).

Tightening:

1. Pull the belts (A) of the backrest cover tighter.

Loosening:

1. Open the buckles (B) by squeezing the retaining lugs together (refer to arrows in Fig. 4-6).
2. Push the belts back a little through the buckle.
3. Close the belt buckles.
4. Retighten the belts gently.

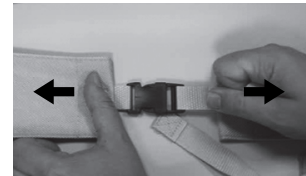
Fig. 4-7

WARNING!

Risk of Injury due to Damaged Backrest Buckles

Check for secure fastening of the backrest buckles each time the product is maintained or reconditioned.

- Check all buckle clips for secure fastening by closing the buckles and pulling on both ends.
- Replace the backrest/backrest buckles if a buckle opens under the applied force.

4.8 Setting Armrest Height

1. Open the belt buckle of the backrest cover which is passed on the inside of the backrest frame.
2. Open the belt buckle on the backrest cover, which is above the armrest.
 -  When adjusting to a lower position the belt beneath the armrest must be opened.
3. Pass this belt on the inside of the backrest frame and close it again.
- 4.

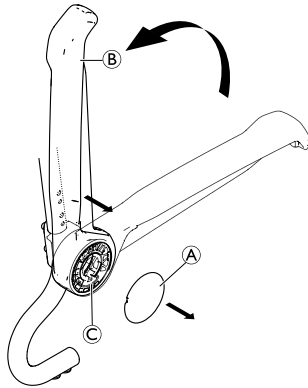
Fig. 4-8

Remove cap (A).

5. Fold up armrest (B) to pull it off the armrest bracket (C).

6.

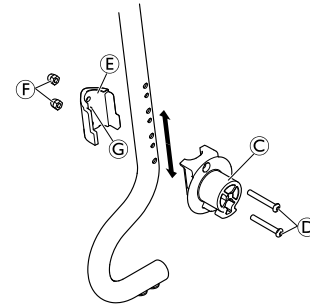
Fig. 4-9

Remove screws (D) from armrest bracket (C) with supplied Allen key.

7. Remove securing clamp (E) with nuts (F) from the inside of the backrest frame.
 -  The nuts are inserted loosely into the securing clamp and may drop out.
8. To reinstall armrest bracket and securing clamp at desired position, insert screws (D) into armrest bracket and guide them through the desired holes in the backrest frame.
 -  The smaller holes on the inside of the backrest frame fix the securing clamp.
9. Place the securing clamp in such way that the fixing (G) of securing clamp engages in smaller hole.
10. Tighten screws (D) with 8 Nm and nuts (F).

11.

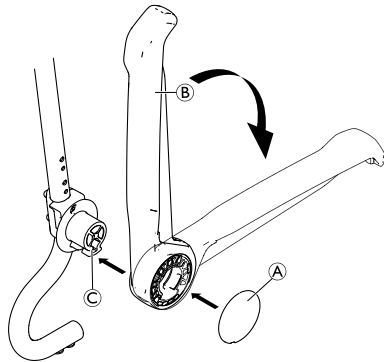
Fig. 4-10

Reinstall armrest **B** on bracket **C**.

12. Fold down armrest.
13. Reinstall cap **A**.
14. Pass the open belt of the backrest cover on the outside of the backrest frame, close belt buckles and tighten all the belts.
15. The height adjustment procedure is the same for the right and left armrest.

 To install accessories, the third belt from the top can be opened and the backrest cover can be passed by the backrest frame from the inside.

4.9 Fitting Footrests

 The right and left footrest are each installed in the same way.

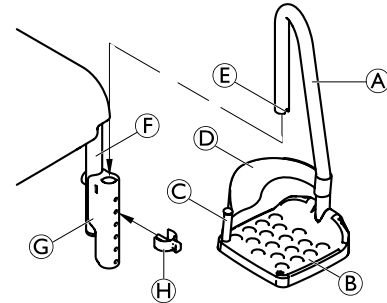
Fig. 4-11

1. Slide footrest **A** with footplate **B** inwards into the guide **G** on seat frame **F** until groove **E** engages in the pin of clip **H**. The footrest should no longer turn to the sides.

Setting Footrest Height

1. Pull footrest **A** up a little.
2. Pull out clip **H** and reinsert it in desired position. You may need to pull the footrest out a little further to do this.
3. Press down the footrest until groove **E** engages in the pin of the clip.
4. Adjust second footrest to desired height the same way.

Removing / Installing Heel Straps

1. Open hook and loop fastener on heel strap **D**.
2. Remove screw **C** from bottom side of the footplate.
3. Remove screw and flexible pin with the heel strap.
4. Installing the heel straps take place in reverse order.

 When installing a heel strap, make sure that the hook and loop fastener faces backwards.

Adjusting Heel Strap

1. Open hook and loop fastener on heel strap ①.
2. Set heel strap to desired length.
3. Close hook and loop fastener again.

4.10 Fitting Electronics Cover

1. Turn both footrests outwards or remove them.
2. Use the hand control to move the product to the highest position, refer to 5.4 *Using Hand Control*, page 24.
- 3.

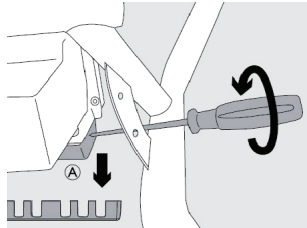
Fig. 4-12

Remove terminal strip ①.

4.

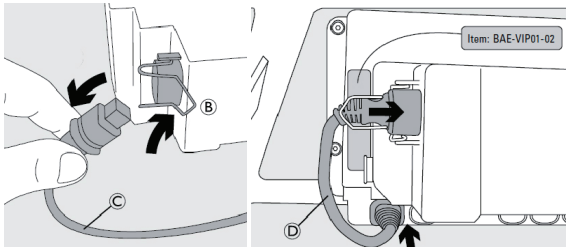
Fig. 4-13

Fold safety bar ② forward.

5. Remove the long connector cable ③ (180 mm) between the battery pack and electronics box.
6. Attach short connector cable ④ (110 mm).
7. Fold the safety bar back again.
8. Reattach the terminal strip.
- 9.

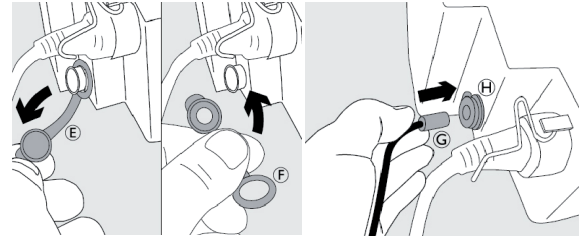
Fig. 4-14

Swap blue plug ⑤ of the charging socket with the plug with the hole ⑥.

10. Insert connector ⑦ on the electronics cover into charger socket ⑧ of the battery pack.

Risk of Damage

– Make sure the connector is positioned correctly.

11.

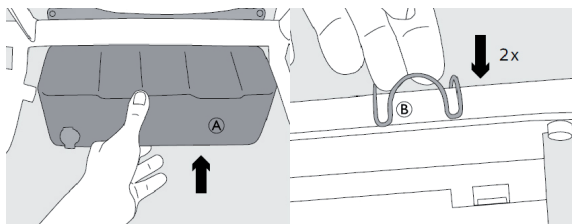
Fig. 4-15

Attach the front section ① to the electronics cover and secure it on the tube with safety clip ②.

12.

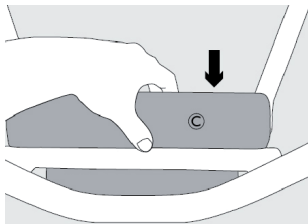
Fig. 4-16

Attach the rear part of the electronics cover ③ from above.

Removal

The electronics cover is removed in reverse order.

4.11 Removing Battery Pack

1. Remove electronics cover, refer to 4.10 *Fitting Electronics Cover*, page 19.

Risk of Damage

– Make sure that the connector between electronics cover and battery pack is disconnected.

2. Remove footrests, refer to 4.9 *Fitting Footrests*, page 18.

3.

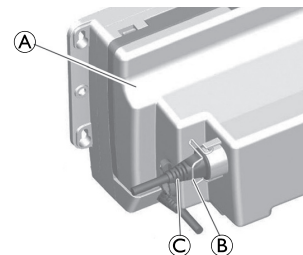
Fig. 4-17

Fold safety bar ② forward and remove connector ③ from battery pack ①.

4.

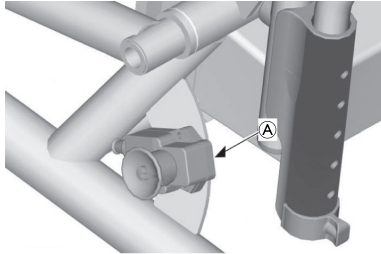
Fig. 4-18

Insert the connector cable in cable holder **A**.

5.

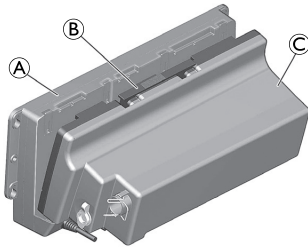
Fig. 4-19

Use one hand to push back unlocking tab **B** on the top of battery pack **C**.

6. With your other hand, lift the battery pack approx. 5 mm and pull it out approx. 10 mm.
7. Remove the battery pack from control box **A** with twist and tilt movement.

4.12 Charging Battery

The product cannot be used while the battery is charging.

The battery can be charged with the electronics cover attached or detached and when the battery is removed.

WARNING!

Risk of Injury or Damage

- Never open the battery pack. Opening the battery pack voids the warranty.
- Only connect the charger to the mains when the battery pack is connected to the charger.

CAUTION!

Risk of Electric Shock

- Only recharge the battery in dry rooms and never in the bathroom.
- With the electronics cover removed after charging, close off the socket on the battery pack with the dust cap.

Risk of Damage

- Only use the supplied charger to charge the battery in the battery pack.
- Make sure that the voltage specifications shown in the charger correspond to those of the mains supply.
- Refer to *6.1 Servicing and Inspection, page 27* for more information about maintaining the battery life when the product is not used for a prolonged time.

1.

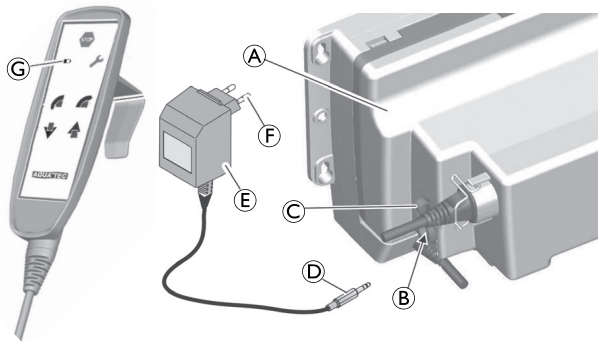
Fig. 4-20

Detach cap © from socket ② on battery pack ①.

When the electronics cover is fitted, the charging socket is to the front not to the side.

2. Fully insert connector ④ of charger ⑤ into socket ② on the battery pack.

3. Insert mains plug ⑥ of the charger into an approved power socket.

Charging takes approx. 5 hours. The indicator lamp ⑦ on the hand control lights up yellow during this time. A trickle charge (pulsed charging) is applied for approx. 3 hours after this time. The indicator lamp on the hand control lights up green during this time. The charger controller terminates the charging process after about 8 hours. The indicator lamp on the hand control turns off. This prevents the battery from being “overcharged”.

4. Disconnect the charger from the power socket.

5. Disconnect the charger connector from the socket on the battery pack.

6. Close off the socket with the dust cap.

5 Usage

5.1 Safety Information

WARNING!

- Only use the product on a flat surface and indoors.
- Observe the specifications on the identification label. Do not overload the product.
- Always perform a visual inspection of the product for external damage before each use.
- Do not use a faulty product.
- Recharge the battery regularly.
- Always use the chest belt when transporting a person.
- Close the chest belt when the product is not use.

WARNING!

Risk of slipping and rolling

- Before use, make sure that the seat is securely fastened to the seat frame.
- Lock all castors with levers using the foot levers before sitting down on the product or standing up from it, as well as in the parking position.

WARNING!

Risk of tipping

- Swivel the footrests to the side or fold up the foot plates before sitting down or standing up.
- Before sitting down or standing up, turn the front castors so that the lock is pointing forwards.
- Never stand on the footrests.
- Never remove the spindle drives.
- Do not lean excessively forwards.

CAUTION!

Risk of burning your skin

- The product components can heat up when exposed to external sources of heat.
- Consider heated up surfaces when the product is exposed to hot water.
 - Do not expose the product to direct sunlight before use.

Risk of Breaking Headrest

- The product must only be moved by using the handle on the backrest, never the headrest.

Risk of Damage

- Do not lift the product using the footrests, armrests or any accessories.

5.2 Turning Armrests

CAUTION!

Risk of Pinching Fingers

- Do not put fingers between armrest and backrest when folding back the armrests.

5.3 Turning Footrests

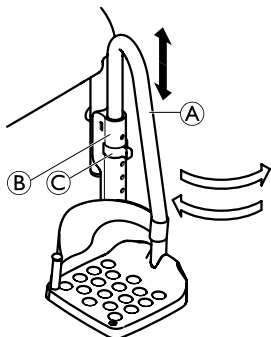
Fig. 5-1

Mounting / Dismounting Position

1. Pull footrest (A) in guide (B) upwards slightly until the groove is no longer engaged in the pin of clip (C).
2. Turn the footrest to the side.

Driving Position

CAUTION!

Risk of Accident due to Loose Footrests

- Loose footrests can cause accidents and injury.
- Secure the footrests in the driving position.

1. Turn footrest (A) inwards.
2. Press down the footrest until the groove engages in the pin of clip (C).

5.4 Using Hand Control

CAUTION!

Risk of Pinching

- Do not put fingers between the seat frame and the front tubular frame of the chassis when tilting or lowering the seat.
- Do not let children play with the product.

The product is moved via the hand control.

The device stops immediately in its current position as soon as the relevant buttons are released.

For safety reasons, the drives have certain blocked ranges:

| Seat Height | Seat Tilt |
|---------------|--|
| 500 to 600 mm | up to 5° forwards up to 35° backwards |
| over 600 mm | no forward movement up to 35° backwards |

If the seat frame is tilted forward, the seat height can be adjusted to a maximum of 600 mm.

If the seat height is set to more than 600 mm, the seat frame can no longer be tilted forward.

! Risk of Damage

Too little space around the device while tilting, lifting or lowering can cause collisions with furnishings.

- Make sure that there is enough space around the device before tilting, lifting or lowering.

! Risk of Damage

If the battery in the battery pack is not sufficiently charged, the indicator lamp **D** on the hand control lights up yellow and an acoustic signal is heard while moving. After the indicator lamp lights up, just five more movement cycles can be carried out. After this, the lifting function of the device is disabled. You can, however, still lower the device.

- Recharge the battery pack immediately after lowering the device.

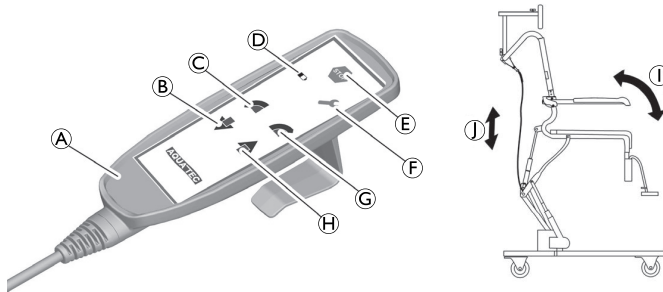
Fig. 5-2

A Hand Control

B Down button

C Forward button

D Indicator lamp

E Emergency stop button

F Service indicator lamp

G Reverse button

H Up button

I Tilt movement

J Height movement

Tilting Seat Frame and Backrest

The seat frame can be tilted backwards approx. 35° and forwards approx. 5°.

1. Press reverse button **G** or forward button **C**.
The seat frame tilts forwards or backwards **I**.

Lifting / Lowering

1. Press up button **H** or down button **B**.
The seat frame moves upwards or downwards **J**.

Emergency Stop

Only press the emergency stop button **E** if the device does not stop immediately when a button is released (e.g. if a button jams). The device stops moving immediately after pressing the emergency stop button.

Emergency Lowering Mechanism

If it is no longer possible to lower the seat frame any further using the hand control, the seat frame can be lowered using the emergency lowering mechanism.

1.

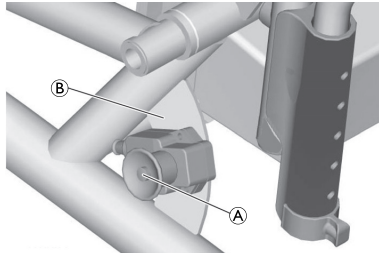
Fig. 5-3

Press button (A) on right side of chassis (B).

5.5 Engaging/Disengaging the Castor Locks


CAUTION!

Risk of Trapping/Squeezing

All castor locks are foot-operated.

– Do not operate the castor locks by hand.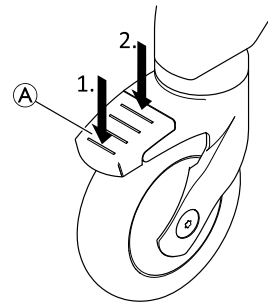

1. To lock the castor locks step on the rear end of the pedal (A) until it engages.

The castor is now locked in position and cannot swivel any longer.

2. To unlock the castor locks step on the front end of the pedal until it disengages.

Castor with blue pedal (option directional lock) engages the swivel lock for easier straight movement.

6 Maintenance

6.1 Servicing and Inspection

CAUTION!

Risk of injury or Damage

- Do not perform any maintenance or service procedures while the product is in use.

The product is maintenance-free, provided the cleaning and safety instructions outlined below are adhered to.

To ensure good performance and safety the following checks should regularly be carried out:

- Check the product and accessories regularly for damage. Do not use the product if defective.
- In case of damage or malfunction it is recommended to contact the provider.
- Ensure that the product is securely assembled and that screws are properly tightened.
- If the product is not used for a prolonged period, charge the battery completely and then raise and lower the seat frame 3 times without load. Repeat this process every 3 months. This helps to maintain the service life of the battery.

WARNING!

Risk of Injury and Damage

If the product is improperly adjusted it can cause accidents and severe injuries.

- The product must be repaired by a qualified technician.
- Damaged parts have to be replaced by a qualified technician.
- Only use original Invacare spare parts.

WARNING!

Risk of Injury or Damage

Incorrectly fitted castors or drives can lead to accidents.

- The castors and drives must be replaced by a qualified technician.

6.2 Cleaning and Disinfection

6.2.1 General safety information

CAUTION!

Risk of contamination

- Take precautions for yourself and use appropriate protective equipment.

**CAUTION!****Risk of electric shock and product damage**

- Switch off the device and disconnect from mains, if applicable.
- When cleaning electronic components consider their protection class regarding water ingress.
- Make sure that no water splashes to the plug or the wall outlet.
- Do not touch the power socket with wet hands.

**IMPORTANT!**

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

6.2.2 Cleaning Intervals**IMPORTANT!**

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
- Clean and disinfect the product
- regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

6.2.3 Cleaning Instructions

The product can be cleaned using commercially available agents.

**Risk of Damage**

- In the sensitive area of the bathroom and toilet, hygiene is particularly important.
- Keep the product clean at all times and wipe it down regularly.

Cleaning in Domestic Environment**Risk of Damage**

- Risk of damage to the product
- Do not disassemble the holders for the accessories.

1. Remove any accessories fitted.
2. Pull the footrests out of the guides.
3. Remove the seat plate and the backrest cover.

4. Remove the side parts.
5. Reinsert the clips in the seat frame immediately so that they are not misplaced.
6. Wipe down the individual parts using a cloth or brush and commercially available cleaning agents and disinfectants.
7. Rinse the parts with warm water.
8. Dry the parts with a cloth.
9. Re-assemble the product, refer to *4 Setup, page 12*.

Cleaning in Clinical Environment

In addition to cleaning as described above, clean the product after the release of each patient, after ending of the period of use or on a minimum monthly basis (depending on which occurs first) at 85 °C for 3 minutes in the autoclave.

CAUTION!

Risk of Burning Your Skin

– Allow the product to cool after cleaning in the autoclave.

Cleaning Backrest Cover

1. Remove the backrest cover.
2. Machine-wash the cover at max. 60 °C using a mild detergent.

Risk of Damage

– Washing at higher temperatures can cause shrinkage.
– Do not tumble dry covers.

Risk of Damage

Excessive cleaning can damage textile.
– We recommend to change the backrest cover every two years for hygiene reasons.

Cleaning Toilet Pan (Optional)

The toilet pan can be cleaned in an autoclave at following temperatures/times:

- 90 °C for 1 minute
- 85 °C for 3 minutes
- 80 °C for 10 minutes.

6.2.4 Disinfection Instructions

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

In Domestic Care

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

In Clinical and Long Term Care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

7 After Use

7.1 Storage and Transport

Risk of Damage

- Store the product in a dry area.
- Do not store the product near heat sources.
- Do not store battery near naked flames.
- Do not expose the product to direct sunlight.
- Refer to the storage and shipping conditions in chapter 9.2 *Environmental Parameters*, page 34.

Transport

The product can be dismantled for transport into the parts of the scope of delivery, refer to 3.3 *Components*, page 10.

1. Disassemble the product: Refer to 4 *Setup*, page 12 by following the assembly steps in reverse order.

7.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see 6 *Maintenance*, page 27 and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7.3 Disposal

WARNING!

Environmental Hazard

Device contains batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste.
- DO NOT throw batteries into a fire.
- Batteries MUST be taken to a proper disposal site. The return is required by law and free of charge.
- Only dispose of discharged batteries.
- Cover terminals of lithium batteries prior to disposal.
- For information on the battery type see battery label or chapter 9 *Technical Data*, page 34.

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

8 Troubleshooting

8.1 Identifying and Repairing Malfunctions

i The following table provides information on malfunctions and their possible causes.

If you cannot correct the malfunction using the remedies named, please contact your provider directly.

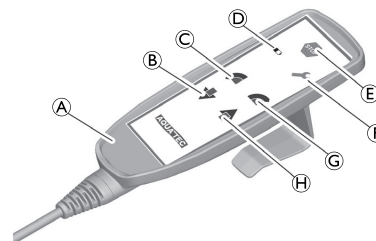
Fig. 8-1

| Malfunction | Possible Cause(s) | Remedy |
|--|--|---|
| An acoustic signal sounds and indicator lamp Ⓓ lights up while the device is moving | Battery almost exhausted | Charge battery, refer to 4.12 <i>Charging Battery</i> , page 21 |
| The service indicator lamp Ⓕ lights up | Hand control faulty | Implement reset (see below table) |
| | Software error | |
| | Emergency lowering mechanism button faulty | |
| | Power failure while moving | |
| | Drive faulty | |
| | Controller faulty | |

| Malfunction | Possible Cause(s) | Remedy |
|--|---|--|
| Emergency lowering mechanism is not working | Battery flat | Charge battery, refer to <i>4.12 Charging Battery, page 21</i> , or use a replacement battery |
| | Cable not fully inserted | Insert cable completely |
| | Connector strip not connected | Connector strip connected (engaged) |
| | Faulty button | Implement reset (see below table) |
| Tilted position of seat frame | The motors were raised to different heights | Hold the down button B until the drives are lowered completely and until you hear the limit switch click into place |
| The seat frame does not move upwards or The seat frame does not move downwards or The seat frame does not tilt | Weight limit exceeded | Max. load 150 kg |
| | Cable not correctly inserted | Insert cable correctly |
| | Connector strip not connected | Connector strip connected (engaged) |
| | Cable incorrectly attached to controller | Motor for tilting always on channel 7 |
| | Connector cable broken | Replace cable |
| | Connector cable not completely inserted | Insert cable completely and close safety bar |
| | Controller faulty | Implement reset (see below table) |
| | Battery flat | Charge battery, refer to <i>4.12 Charging Battery, page 21</i> , or use a replacement battery |
| Hand control cable is not inserted correctly | Insert cable completely | |

| Malfunction | Possible Cause(s) | Remedy |
|-------------------------|----------------------------------|---|
| | Hand control faulty | Implement reset (see below table) or Replace hand control |
| Battery does not charge | Faulty battery | Replace battery |
| | Connector not correctly inserted | Check that all connectors are positioned correctly |

Implementing Reset

1. Hold down the following three buttons simultaneously for about three seconds:

- Emergency stop button (E),
- Forward button (C) and
- Up button (H).

An acoustic signal is heard while the three buttons are held down.

2. When the service indicator lamp (F) turns off, release the buttons. The device is ready to use again.
3. Then move the drives for adjusting the height to the lowest position until you hear the drives switch off.
4. Move the tilt drive to the rear end position until you hear the drive switch off. (The red LED flashes and an acoustic signal is heard).

If the fault recurs after the reset, replace the device or the faulty parts and send them to your distributor (addresses see last page).

9 Technical Data

9.1 Dimensions and Weights

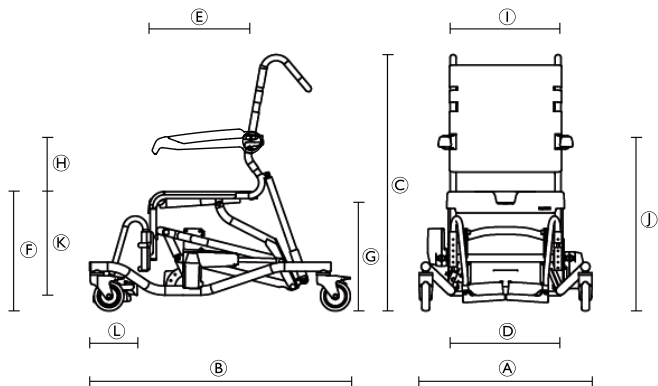
Fig. 9-1

| | | |
|-----|---|----------------|
| (A) | Total width | 730 mm |
| (B) | Depth (without footrests) | 1010 mm |
| (C) | Height (without headrest) | 1070 – 1620 mm |
| (D) | Seat width | 480 mm |
| (E) | Seat depth | 450 mm |
| (F) | Seat height | 500 – 1050 mm |
| (G) | Measure the distance between the floor and the bracket for the commode pan. | 420 – 970 mm |

| | | |
|-----|----------------------------|---------------|
| (H) | Height, seat to armrest | 230 – 270 mm |
| (I) | Width between the armrests | 475 mm |
| (J) | Height to armrests | 730 – 1320 mm |
| (K) | Height, footrests to seat | 345 – 465 mm |
| (L) | Footplate depth | 155 mm |
| (M) | Width hygiene recess | 205 mm |
| (N) | Depth hygiene recess | 320 mm |
| | Reversing width | 2690 mm |
| | Turning diameter | 1290 mm |
| | Height adjustment | stepless |
| | Castor diameter | 5" (127 mm) |
| | Weight | approx. 39 kg |
| | Load capacity | 150 kg |

9.2 Environmental Parameters

| | |
|--|-----------------------------|
| Operating conditions | |
| Ambient temperature | 5 - 40 °C |
| Relative humidity | 20 % - 70 %, non-condensing |
| Atmospheric pressure | 795 - 1060 hPa |
| Storage and shipping conditions | |
| Ambient temperature | 0 - 40 °C |

| | |
|----------------------|-----------------------------|
| Relative humidity | 30 % - 70 %, non-condensing |
| Atmospheric pressure | 795 - 1060 hPa |

9.3 Electronic Data

| | |
|---|-------------------------------|
| Operating voltage | 24 V |
| Rated input power | 170 VA |
| Battery type | Lead acid |
| Typical operating time | 40 cycles |
| Charger input | 230 – 240 V AC, 50 – 60 Hz |
| Charger output | 24 V DC, 12 VA |
| Charge time from low-voltage warning | approx. 5 – 6 h |
| Type of protection: drive, battery pack and electronics housing | IP X6 ¹ |
| Type of protection: charger | IP 40 ² |
| Type of protection: entire device | IP X6 ¹ |
| Sound power level | 52 dB(A) |

- 1 Protected against the penetration of water in the event of strong jets of water.
- 2 Protected from solid foreign bodies measuring 1.0 mm or more, not protected against the penetration of water.

9.4 Materials

| | |
|----------------|---------------------------------------|
| Frame | Stainless steel, powder-coated |
| Seat plate | Polypropylene |
| Backrest cover | Nylon, PVC-coated |
| Footrests | Polypropylene, fiber-glass reinforced |
| Armrest | Polyamid, fiber-glass reinforced |
| Castors | Plastics and stainless steel |

All product components are corrosion resistant and do not contain natural rubber latex.

10 Electromagnetic compatibility (EMC)

10.1 General EMC information

Medical Electrical Equipment needs to be installed and used according to the EMC information in this manual.

This product has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2 for Class B equipment.

Portable and mobile RF communications equipment can affect the operation of this product.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standard. To determine if the emission from this product is causing the interference, run and stop running this product. If the interference with the other device operation stops, then this product is causing the interference. In such rare cases, interference may be reduced or corrected by the following:

- Reposition, relocate, or increase the separation between the devices.

10.2 Electromagnetic emission

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

| Emissions test | Compliance | Electromagnetic environment - guidance |
|--------------------------|------------|--|
| RF emissions CISPR 11 | Group I | This product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. |
| RF emissions CISPR 11 | Class B | This product is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |

| | | |
|--|----------|--|
| Harmonic emissions IEC 61000-3-2 | Class A | |
| Voltage fluctuations /flicker emissions IEC 61000-3-3 | Complies | |

10.3 Electromagnetic Immunity

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

| Immunity test | Test / Compliance level | Electromagnetic environment – guidance |
|--|---|--|
| Electrostatic discharge (ESD) IEC 61000-4-2 | ± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %. |
| Electrostatic transient / burst IEC 61000-4-4 | ± 2 kV for power supply lines; 100 kHz repetition frequency | Mains power quality should be that of a typical commercial or hospital environment. |
| Surge IEC 61000-4-5 | ± 1 kV line to line | Mains power quality should be that of a typical commercial or hospital environment. |

| Immunity test | Test / Compliance level | Electromagnetic environment – guidance |
|--|--|--|
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 | $< 0\% U_T$ for 0,5 cycle at 45° steps $0\% U_T$ for 1 cycles $70\% U_T$ for 25 / 30 cycles $< 5\% U_T$ for 250 / 300 cycles | Mains power quality should be that of a typical commercial or hospital environment. If the user of this product requires continued operation during power mains interruptions, it is recommended that the product is powered from an un-interruptible power supply or a battery. U_T is the a. c. mains voltage prior to application of the test level. |
| Power frequency (50/60 Hz) magnetic field IEC 61000-4-8 | 30 A/m | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |
| Conducted RF IEC 61000-4-6 | $3 V$ $150 \text{ kHz to } 80 \text{ Mhz}$ $6 V$ in ISM & amateur radio bands | Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which this product is used exceeds the applicable RF compliance level above, this product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating this product. Interference may occur in the vicinity of equipment marked with the following symbol:  |

| Immunity test | Test / Compliance level | Electromagnetic environment – guidance |
|------------------------------|---|---|
| Radiated RF IEC 61000-4-3 | 20 V/m 80 Mhz to 2,5 GHz 385 MHz - 5785 MHz test specifications for immunity to RF wireless communication equipment refer to table 9 of IEC 60601-1-2:2014 | Portable and mobile RF communications equipment should be used no closer than 30 cm to any part of this product including cables. |

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Invacare Limited
Invacare Austria GmbH
Invacare GmbH