Invacare[®] Orion Series

Orion^{METRO}, Orion^{PRO}



en Scooter Service Manual



PROVIDER: Keep this manual. The procedures in this manual MUST be performed by a qualified technician.

Yes, you can:

© 2020 Invacare Corporation All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by[™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. Making Life's Experiences Possible is a registered trademark in the U.S.A.

Contents

	I	
1 Gen	eral	4
1.1	Introduction	4
1.2		4
1.3		4
-		
1.4		4
1.5	Images in This Manual	4
2 6 . 4 .	ty	5
z sale		-
2.1	Safety information.	5
2.2	Safety and Fitting Instructions	5
		_
3 Hygi	ene	7
3.1	Handling of Returned Used Products	7
-	-	
4 Setu	p	8
4.1	Operating console (LED version)	8
		8
		-
4.1		9
4.2		9
4.2	.1 Entering setup (LCD Display) 1	0
4.2		
4.2		
	4 Cotting wheel diameter	1
4.2	.4 Setting wheel diameter 1	T
5 Serv	ice	2
5.1	General safety information on installation work 1	
5.2	Troubleshooting	2
5.2	.1 Operational faults 1	2
5.2	•	
5.2		
-		
5.3	Tightening Torques 1	
5.4	Service Plan (Once a Year)1	4
5.5	Shrouds	5
5.5	.1 Removing shroud 1	5
5.5		
5.6		
5.6		5
5.6	.2 Replacing Circuit Board 1	6
5.6	.3 Replacing Potentiometer1	7
5.6		
	Steering	
5.7		
5.7		0
5.7	.3 Replacing Steering Bar	1
5.7	.4 Replacing curve control device	1
	Lighting unit	
	.1 Replacing headlight	
5.8		2
5.8		3
5.8	.4 Replacing rear lights/direction indicators	4
5.9	Wheels	
5.9		-
5.5	Version)	E
5.9	1 6	
5.9		6
5.9	.4 Removing Front Shock Absorber (3–Wheel Orion ^{PRO}	
	only)	6
5.9		
5.9		
5.9	0	
5.9		
5.10	Brakes	8
5.1		
5.1		
5.11	Drive Components	
5.1		
5.1	1.2 Replacing carbon brushes	1
5.12	Seat system	
5.1		
5.1		
	2.2 Replacing seat support tube	

1 General

1.1 Introduction

This document contains important information about assembly, adjustment and advanced maintenance of the product. To ensure safety when handling the product, read this document and the user manual carefully and follow the safety instructions.

Find the user manual on Invacare's website or contact your Invacare representative. See addresses at the end of this document.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

For pre-sale and user information, see the user manual.

For more information about the product, for example product safety notices and product recalls, contact your Invacare representative. See addresses at the end of this document.

1.2 General Information

Service and maintenance work must be carried out taking this document into account.

It is imperative that you observe safety information.

Information about operation or about general maintenance and care work on the product should be taken from service manual.

Assembly of accessories might not be described in this document. Refer to the manual delivered with the accessory. Additional manuals can be ordered from Invacare. See addresses at the end of this document.

You can find information about ordering spare parts in the spare parts catalogue.

Spare parts must match original Invacare parts. Only use spare parts which have been approved by Invacare.

The product may only be maintained and overhauled by qualified personnel.

The minimum requirement for service technicians is suitable training, such as in the cycle or orthopedic mechanics fields, or sufficiently long-term job experience. Experience in the use of electrical measuring equipment (multimeters) is also a requirement. Special Invacare training is recommended.

Alterations to the mobility device which occur as a result of incorrectly or improperly executed maintenance or overhaul work lead to the exclusion of all liability on the side of Invacare.

If you have any problems or questions contact your provider.

1.3 Notes on Shipping

- If the mobility device has to be shipped back to the manufacturer for major repairs, you should always use the original packaging for transport.
- Please attach a precise description of the fault.

1.4 Symbols in This Manual

Symbols and signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.



WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

n Tips

Gives useful tips, recommendations and information for efficient, trouble-free use.

Identifies required tools, components and items which are needed to carry out certain work.

1.5 Images in This Manual

The detailed images in this manual are given marks to identify various components. Component marks in text and operational instructions always relate to the image directly above.

2 Safety

2.1 Safety information

WARNING!

The procedures in this service manual, must be performed by a specialized dealer or qualified service technician.

- Do not handle this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, installation manuals or instruction sheets supplied with this product or optional equipment.

2.2 Safety and Fitting Instructions

These safety instructions are intended to prevent accidents at work, and it is imperative that they are observed.

Before any inspection or repair work

- Read and observe this repair manual and the associated user manual.
- Observe the minimum requirements for carrying out the work (see 1.2 General Information, page 4).

Personal Safety Equipment

Safety shoes

The mobility device, and some of its components, are very heavy. These parts can result in injuries to the feet if they are allowed to drop.

• Wear standardized safety shoes during all work.

Eye protection

It is possible that battery acid can be discharged when working on defective batteries or when handling batteries improperly.

 Always wear eye protection when working on any defective or possibly defective batteries.

Safety gloves

It is possible that battery acid can be discharged when working on defective batteries or when handling batteries improperly.

• Always wear acid-proof safety gloves when working on any defective or possibly defective batteries.

General Safety Information and Information About Fitting / Removal

Â

Risk of Death, Serious Injury, or Damage

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury, or damage. Mobility device occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the mobility device.

DO NOT smoke while using this mobility device.



WARNING!

DANGER!

Risk of Serious Injury or Damage

Storing or using the mobility device near open flame or combustible products can result in serious injury or damage.

 Avoid storing or using the mobility device near open flame or combustible products.



CAUTION! Risk of crushing

Various components such as the drive unit, batteries, seat etc are very heavy. This results in injury hazards to your hands.

Note the high weight of some components.
 This applies especially to the removal of drive units, batteries and the seat.

CAUTION!

Injury hazard if the mobility device starts moving unintentionally during repair work

- Switch the power supply off (ON/OFF key).
- Engage the drive.
- Before lifting up, secure the mobility device by using chocks to block the wheels.

CAUTION!



Fire and burn hazard due to electrical short-circuit

- The mobility device must be completely switched off before removal of voltage-carrying components! To do this, remove the batteries.
- Avoid short-circuiting the contacts when carrying out measurements on voltage-carrying components.



Risk of burns from hot surfaces on the motor – Allow the motors to cool down before commencing work on them.

CAUTION!

Injury hazard and risk of damage to mobility device due to improper or incomplete maintenance work

- Use only undamaged tools in good condition.
- Some moving parts are mounted in sockets with PTFE coating (Teflon[™]). Never grease these sockets!
- Never use "normal" nuts instead of self-locking nuts.
- Always use correctly-dimensioned washers and spacers.
- When reassembling, always replace any cable ties which were cut during dismantling.
- After completing your work / before renewed start-up of the mobility device, check all connections for tight fitting.
- After completing your work / before renewed start-up of the mobility device, check all parts for correct locking.
- Only operate the mobility device with the approved tyre pressures (see technical data).
- Check all electrical components for correct function. Note that incorrect polarity can result in damage to the control system.
- Always carry out a trial run at the end of your work.



CAUTION!

Risk of injury and damage to property, if the maximum speed reduction on a wheelchair with a lifter does not function correctly The wheelchair's control unit must reduce the maximum possible speed as soon as the lifter is raised.

 Test the maximum speed reduction for correct function after any maintenance work or modifications to the wheelchair.

CAUTION!

Any changes to the drive program can affect the driving characteristics and the tipping stability of the mobility device

- Changes to the drive program may only be carried out by trained Invacare providers.
- Invacare supplies all mobility devices with a standard drive program ex-works. Invacare can only give a warranty for safe mobility device driving behavior - especially tipping stability - for this standard drive program.
- Mark all current settings for the mobility device (seat, armrests, backrest etc.), and the associated cable connecting plugs, before dismantling. This makes reassembly easier. All plugs are fitted with mechanical locks which prevent release of the connecting plugs during operation. To release the connecting plugs the safety locks must be pressed in. When reassembling ensure that these safety locks are correctly engaged.

3 Hygiene

3.1 Handling of Returned Used Products

When reconditioning or repairing returned mobility devices:

- Take precautions for yourself and the product.
- Use protection equipment as specified locally.

Before Transport (According to Biological Agents Ordinance)

Treat product according to following process steps:

Process Step	Component	Application	Conditioning technique	Work Station
Manual cleaning	Surface of used device	Before repair or reconditioning	Use saturated towel to apply cleaning detergent and remove residues after impact.	Cleaning and disinfection
Disinfection	Surface of used device	Before repair or reconditioning	Use saturated disinfectant wipes and clean* the device surface.	Cleaning and disinfection

*Invacare uses detergent "Nücosept special" 1.5% in water ml/ml

Disinfection Tools

- Disposable wipes (fleece)
- Brushes to clean areas difficult to access


Further Information

 $\mathring{\parallel}$ For more information contact your Invacare service department.

4 Setup

4.1 Operating console (LED version)

Arrangement

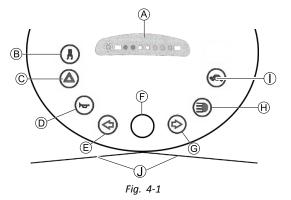

(A)	Status display
₿	Switching on/off curve control (reduction of speed when driving in a curve)
\bigcirc	Hazard lights
D	Horn
E	Left-hand direction indicator (switches itself off automatically after 30 seconds)
Ð	Speed control dial
G	Right-hand direction indicator (switches itself off automatically after 30 seconds)
$^{(\!\!\!\!\!)}$	Lighting
	Low speed mode
	Drive lever

Status Display



Fig. 4-2

The ON/OFF diode (1) is used as a fault display (status display). It will flash if there is a problem with the scooter. The number of flashes indicates the type of error. Refer to 5.2.3 Error Codes And Diagnostic Codes, page 13.

Battery Indication Gauge

Low battery indication: Every time the scooter is activated or at work when battery capacity is lower than 25%, the electronic system will beep three times.

Overdischarge protection: after a certain drive time on reserve battery power the electronic system switches the drive off automatically and brings the scooter to a standstill. If you do not drive your scooter for a while the batteries will "recuperate" and allow a further, but short, journey. However, after a very brief journey the battery reserve symbol will illuminate again and the electronic system will beep three times. This procedure leads to battery damage and should be avoided if possible!

Battery capacity: <25 %	Reduced driving range. Recharge the batteries at the end of your journey.
Battery capacity: <20 %	Battery reserve = severely restricted driving range. Recharge batteries immediately!

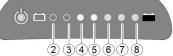


Fig. 4-3

(2)	(3)	(4)	(5)	(6)	(7)	(8)	Battery capacity
Þ	¢	¢	¢	¢	¢	¢	>80 %
₽	¢	¢	₽	₽	₽		<80 %
₽	¢	¢	₽	₽			<65 %
₽	¢	¢	₽				<50 %
₽	¢	¢					<35 %
₽	¢						<25 %
₿							<20 %

4.1.1 Entering setup (LED Display)

- 1. Turn key to switch mobility device off.
- 2.



Press and hold keys.

- Turn key to switch mobility device on. Mobility device enters setup mode after two seconds. LED 2 and LED 8 will flash.
- 4.




Release keys to enter setup settings.

1604202-C

4.1.2 Setting Battery Gauge

- $\underline{\mathring{l}}$ This chapter applies for mobility devices with LED display.
- Setting battery gauge is mandatory when
- retrofitting operating console.

Adjust the parameter in battery gauge to suit different types and sizes of batteries.

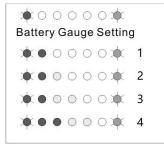

AGM battery		Gel battery		
(1) small size batteries (≤ 50 Ah, default)	(2) big size batteries (> 50 Ah)	(3) small size batteries (≤ 50 Ah)	(4) big size batteries (> 50 Ah)	

1.




Press key to change settings.

2. Save settings by waiting for 10 seconds or pressing any other keys except keys above.

4.2 Operating console (LCD version)


Arrangement

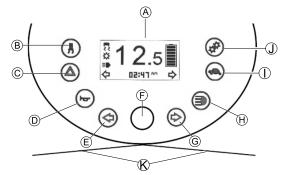

A	Status display
₿	Switching on/off curve control (reduction of speed when driving in a curve)
\odot	Hazard lights
\bigcirc	Horn
®	Left-hand direction indicator (switches itself off automatically after 30 seconds)
Þ	Speed control dial

- © Right-hand direction indicator (switches itself off automatically after 30 seconds)
- (H) Lighting
- ① Low speed mode
- ① Setting
- (K) Drive lever

Status display

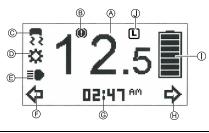

A	Speed indication
₿	Fault indication
©	Curve control indication
D	Maintenance indication ¹
Œ	Head light indication
F	Left turn indication
G	Settings shown: ODO, TRIP, TEMP, TIME
(\boldsymbol{H})	Right turn indication
()	Battery status
1	Low speed selection indication

1If this symbol starts flashing for one minute every time the scooter is switched on, contact your provider.

Battery indication gauge

- Low battery indication: Every time the scooter is activated or at work when battery capacity is lower than 25%, the electronic system will beep three times.
- Overdischarge protection: after a certain drive time on reserve battery power the electronic system switches the drive off automatically and brings the scooter to a standstill. If you do not drive your scooter for a while the batteries will "recuperate" and allow a further, but short, journey. However, after a very brief journey the battery reserve symbol will illuminate again and the electronic system will beep three times. This procedure leads to battery damage and should be avoided if possible!

Battery capacity: <25%	Reduced driving range. Recharge the batteries at the end of your journey.
Battery capacity: <20%	Battery reserve = severely restricted driving range. Recharge batteries immediately!

>80%	<80%	<65%	<50%	<35%	<25%	<20%


4.2.1 Entering setup (LCD Display)

- This chapter applies for mobility devices with LCD ĵ display.
- Turn key to switch mobility device off. 1.
- 2.



Press and hold keys.


3. Turn key to switch mobility device on. Mobility device enters setup mode after two seconds.

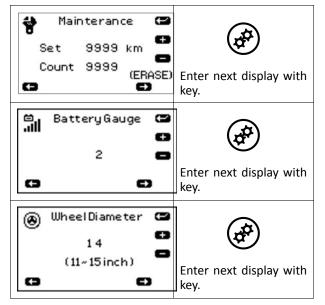

4.2.2 Setting counter

This chapter applies for mobility devices with LCD ĵ display.

Enter setup settings as described in 4.2.1 Entering 1. setup (LCD Display), page 10.

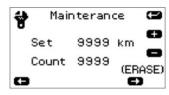

Default setting is 0. You can set a km value ñ as service interval.

If counter value is greater than set value, symbol will flash for one minute, when key is switched on.

+	0 (OFF)+ 500 + 1000 +1500 + 2000 + 9500 + 9999 (max)
-	9999 - 9500 - 9000 - 8500 - 8000 - 500 - 0 (OFF)
$\widehat{}$	

2.

3.

4.

5.



Press and hold key for 2 sec. to erase counter.



Press key to increase value.



Press key to decrease value.




Press key to save and enter next page.

Setting Battery Gauge 4.2.3

- This chapter applies for mobility devices with LCD ĭ display.
- Setting battery gauge is mandatory when ĵ retrofitting operating console.

Adjust the parameter in battery gauge to suit different types and sizes of batteries.

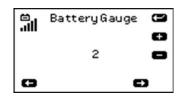

AGM battery		Gel battery		
(1) small size batteries (≤ 50 Ah, default)	(2) big size batteries (> 50 Ah)	(3) small size batteries (≤ 50 Ah)	(4) big size batteries (> 50 Ah)	





Press key to increase value.

2.

Press key to decrease value.

3.


(*)

Press key to save and enter next page.

4.2.4 Setting wheel diameter

- $\label{eq:linear} \overset{\circ}{\underline{l}} \qquad \mbox{This chapter applies to mobility devices with LCD} \\ \mbox{display.}$
- $\label{eq:setting_setting_setting} \overset{\circ}{\underline{l}} \qquad \mbox{Setting wheel diameter is mandatory when} \\ retrofitting operating console.$

Set value according to wheel diameter 11", 12", 13", 14" or 15" for correct display of speed.

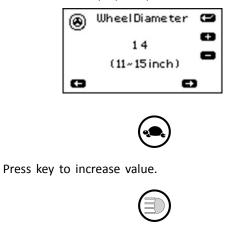

Press key to decrease value. 3.

1.

2.



Press key to save and leave setup settings.

5 Service

5.1 General safety information on installation work

Risk of damage to mobility device Collisions can be caused if shim rings are removed from the drive wheels during installation work. Shim rings are frequently placed between drive shaft and wheel hub to compensate tolerances. Collisions can be caused if these shim rings are removed and not re-installed.

- Install all shim rings in exactly the same positions they were in before dismantling.

Problem **Other Symptoms Possible Cause** Solution Documentation Scooter will not start Status display on Batteries defective **Replace** batteries See user manual operating console Completely Pre-charge batteries does not illuminate discharged battery Power supply to Check main fuse See user manual for operating console main fuse position interrupted Check cable Operating console See Replacing power defective between modules module for loose connections or damage Replace operating See 5.6.1 Replacing Operating Console, console page 15 Status display on Various causes Assess error code See controls user operating console manual flashing Drive motor(s) Scooter judders in drive mode None Replace motor(s) SeeReplacing drive defective unit Drive lever See 5.6.3 Replacing Replace potentiometer potentiometer Potentiometer, page defective 17 None Batteries not being charged Batteries defective **Replace batteries** See user manual LEDs flashing on Charger defective See charger user Replace charger charger manual Scooter runs too slowly None Operating console Replace operating See 5.6.1 Replacing defective console Operating Console, page 15 Batteries defective **Replace** batteries See user manual Scooter does not reduce None Curve control device Adjust curve control speed on bends for bend travel not device adjusted Curve control device Replace curve See 5.7.4 Replacing defective control device curve control device, page 21

5.2.2 Drive Fault Diagnosis

5.2 Troubleshooting

5.2.1 Operational faults

Proceed as follows if you have any problems:

- 1. First assess the possible cause of the problem using the following table.
- 2. Check operating console status display. Evaluate flash code.
- 3. Carry out necessary checks and repairs as recommended in the following table.

5.2.3 Error Codes And Diagnostic Codes

Flash code	Fault	Consequence for the scooter	Comments
1	Batteries must be charged	Continues to drive	• The batteries are discharged. Charge the batteries as soon as possible.
2	Battery voltage too low	Stops driving	 The batteries are depleted. Charge batteries. If you switch the scooter off for a few minutes, the batteries can often recuperate to such a stage that a short journey is still possible. You should only do this in an emergency, however, because this causes the batteries to become excessively discharged.
3	Battery voltage too high	Stops driving	 The battery voltage is too high. If the battery charger is connected, disconnect it from the scooter. The electronic system charges the batteries when running downhill and when braking. This fault is caused when the battery voltage becomes too high during this process. Switch the scooter off and on again.
4	Power time exceeded	Stops driving	 The maximum current was exceeded over too long a period, probably because the motor was overloaded or has been working against an immovable resistance. Switch the scooter off, wait a few minutes and then switch on again. The electronic system has determined a motor short-circuit. Check the wiring harness for short-circuit and check the motor. Contact your Invacare provider.
5	Brake failure	Stops driving	 Ensure that the disengaging lever is in the engaged position. There is a defect in the braking coil or in the cabling. Check the magnetic brake and cabling for open or short-circuited circuitry. Contact your Invacare provider.
6	No neutral position when switching scooter on.	Stops driving	 Drive lever is not in neutral when the keyswitch was turned. Put the drive lever in neutral, turn the power off and then turn on again. It may be necessary to replace the drive lever. Contact your Invacare provider.
	Circuit board failure	Stops driving	 Circuit board got damaged while connecting or disconnecting the charger to the charger socket. Test resistance of drive lever to eliminate a defective lever. It may be necessary to replace the circuit board or drive lever. Contact your Invacare provider.
7	Fault in speed potentiometer	Stops driving	 The drive lever controls could be faulty or incorrectly connected. Check the cabling for open or short-circuited circuitry. Potentiometer is not correctly adjusted and must be replaced. Contact your Invacare provider.
8	Motor voltage error	Stops driving	• The motor or its cabling is defective. Check the cabling for open or short-circuited circuitry.
9	Miscellaneous internal fault	Stops driving	Contact your Invacare provider.
10	Push/freewheel mode error	Stops moving	• The scooter has exceeded the permissible maximum speed during pushing or freewheeling. Switch the electronics system off and on again.

5.3 Tightening Torques

CAUTION!

Risk of damage to mobility device due to improperly tightened screws, nuts or plastic connections.

- Always tighten screws, nuts etc. to the stated tightening torque.
- Only tighten screws or nuts which are not listed here fingertight.

The tightening torques stated in the following list are based on the thread diameter for the nuts and bolts for which no specific values have been determined. All values assume dry and de-greased threads.

Thread	Tightening Torque in Nm ±10 %
M4	3 Nm
M5	6 Nm
M6	10 Nm
M8	25 Nm
M10	49 Nm
M12	80 Nm
M14	120 Nm
M16	180 Nm

5.4 Service Plan (Once a Year)

Component Check		Remedy	Notes	\checkmark
Seat	Check welded seams, fixings and upholstery	Tighten screws, replace parts if damaged		
	Check seat suspension	Grease the spring generously, replace parts if damaged	See Replacing seat suspension/spring	
Frames (Chassis) / Battery Mounting	Check fixings, welded seams and battery mounting	Tighten screws, replace parts if damaged		
	Check battery fixing straps			
Wheel Suspension and Wheels	Check drive wheels for tight fit	Tighten hub nuts, replace if necessary		
	Check front wheels for tight fit, float and side play	Adjust / replace		
	Check pneumatic tire	Repair or replace if damaged	See user manual	
	Check tracking and steering link	Tighten screws		
		Check tracking and steering		
Brake	Check brake function in pushing mode	Adjust or replace brake		
Shock Absorber	Check shock absorber	Adjust or replace shock absorber		
Drive Units, Coupling Mechanism	Check functions in drive and push modes	Replace motor if necessary		
	Check clutch mechanism	Tighten screws / nuts, adjust or replace		
Lighting	Check function			
	Check cable/plug connections	Replace lightbulbs or cable		

Component	Check	Remedy	Notes	\checkmark
Batteries	Check batteries for damage	Replace batteries	See user manual	
	Check battery voltage	Charge batteries	See user manual	
	Check contacts and terminals	Clean contacts and terminals		
Drive Controls	Check status display (flashing)	Evaluate flash code		
	Check fixing	Tighten or replace fixing		
	Check cable, connecting plug	Replace cable, connecting plug		
	Check drive lever function	Replace drive lever		
	Check power supply	Replace cable, connecting plug or console		
Drive Program	Check drive program version. New version available?	Update software		
Curve Control in Bends	Check cable and connecting plug	Adjust or replace sensor on curve control device		
	Check function			
Screws Check screws for tight fit		Tighten screws if necessary		

5.5 Shrouds

5.5.1 **Removing shroud**



CAUTION! Risk of damage to shroud

- Do not pull shroud straight up to remove.

- Follow instructions below.

1.



Place one hand at the front, the other at the rear of the shroud A.

- Lift shroud up from front. 2.
- 3.



Place hands on left and right side of shroud. Twist and lift to remove shroud completely. 4.

5.5.2 **Replacing front shroud**

- 10 mm socket wrench
 - Phillips screwdriver

1. Remove shroud. See 5.5.1 Removing shroud, page 15. 2.



Remove four caps A.

- 3. Loosen and remove screws B.
- Take off front shroud. 4.
- 5. Install parts in reverse order.

5.6 Controls

5.6.1 Replacing Operating Console

CAUTION!

- Risk of burns if power cable is shorted - Turn off the power completely before removing any power supply components of mobility device. Therefore take out batteries.
- Avoid bridging of contacts during measurements on live electrical components.


CAUTION!

- - Risk of damage to operating console/circuit


5.

- board by instant high current/voltage
- Unplug battery cable before
- removing/installing operating console.
- Make sure all pins are correctly connected.
- **Risk of Damaging Circuit Board by Electrostatic** Discharge
 - Handle circuit boards only in Electrostatic Protected Areas (EPA).
- The plugs on the operating console cannot be ື່ງໃ wrongly connected because all plugs have a different size and only fit in one socket.
- łĭ Phillips screwdriver •
- Setting battery gauge is mandatory when ຶ່ງໃ retrofitting operating console.
- Setting wheel diameter is mandatory when j retrofitting operating console.
- Remove seat. 1.
- 2. Remove shroud.
- 3.


4.

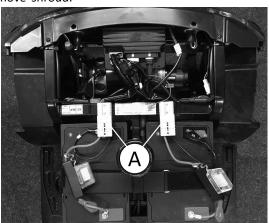

Disconnect battery cable A.

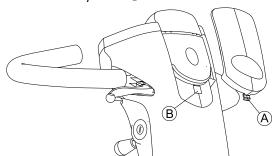

Remove top shroud (A) from steering column (B).

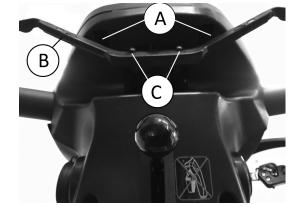

Remove screws (A) below operating console.

- Remove operating console and fold upwards. 6.
- 7. Remove screws © on drive lever [®] and remove drive lever. 8.
 - LED LCD

Fig. 5-1

Fig. 5-2

Disconnect plug from operating console circuit board.

- 9. Replace operating console.
- 10. Connect plug with circuit board on new operating console.
- 11. Install parts in reverse order.
- 12. Test all functions (trial run).

Replacing Circuit Board 5.6.2

CAUTION!

- Risk of burning if power cable is shorted
 - Turn off power completely before removing any power supply components of mobility device. Therefore take out batteries.
 - Avoid bridging of contacts during measurements on live electrical components.


CAUTION!

Risk of damage to operating console/circuit board by instant high current/voltage

- Unplug battery cable before
 - removing/installing circuit board.
- Make sure all pins are correctly connected.
- **Risk of Damaging Circuit Board by Electrostatic** Discharge
 - Handle circuit boards only in Electrostatic Protected Areas (EPA).

1. Unplug battery connector.

1. 2.

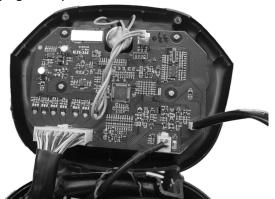

Remove operating console as described in chapter 5.6.1 Replacing Operating Console, page 15.

- 3. Replace circuit board.
- 4. Make sure all pins are correctly connected.
- 5. Plug in battery connectors.
- 6. Install parts in reverse order.
- 7. Test circuit board function.

5.6.3 Replacing Potentiometer

CAUTION!

- Risk of burning if power cable is shorted
 Turn off the power completely before removing any power supply components of the mobility device. Therefore take out batteries.
- Avoid bridging of contacts during measurements on live electrical components.

\triangle

CAUTION!

Risk of damage to operating console by instant high current/voltage

- Unplug battery cable before
- removing/installing operating console.
- Make sure all pins are correctly connected.



CAUTION! Risk of accident

Wrong potentiometer setting can result in dangerous driving situations.


 Connect potentiometer to drive lever mounting.

1	Risk of Damaging Circuit Board by Electrostatic
:	Discharge
	 Handle circuit boards only in Electrostatic

Protected Areas (EPA).

IY •	Phillips	screwdriver
------	----------	-------------

- 2 mm Allen key
 - Digital multimeter
- Before replacing the potentiometer check resistance of it. In some cases a defective circuit board produces the same failures.
- 1. Remove seat.
- 2. Remove shroud.

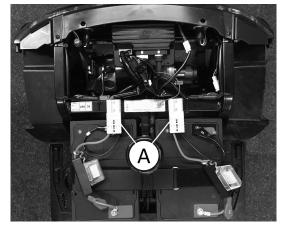


Disconnect battery cable (A).


3.

4.

5.

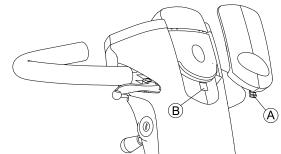


Remove top shroud (A) from steering column (B).

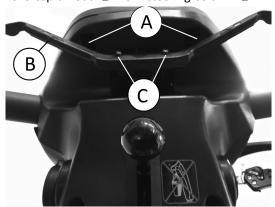

Remove screws (A) below operating console.

- 6. Remove operating console and fold upwards.
- 7. Remove screws © on drive lever ® and remove drive lever.


8.

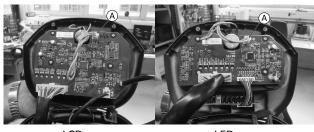

LCD Fig. 5-3


LED Fig. 5-4

Disconnect potentiometer plug $\textcircled{\sc b}$ from operating console.

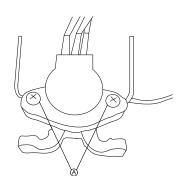


- Remove screws (A) on potentiometer.
- 10. Replace potentiometer.
- 11.

9.

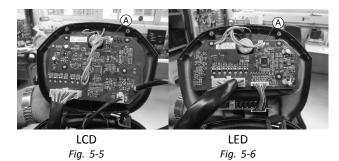

Reconnect potentiometer plug B with operating console.

12.

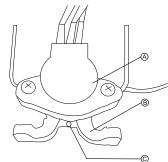

Connect potentiometer (A) and drive lever mounting (B):

- Tighten screw ©.
- 13. Install parts in reverse order.
- 14. Test all functions (trial run).

5.6.4 Replacing Power Module

Risk of accident

CAUTION!

When parking mobility device on antitippers it is no longer slowed by motor brake. Mobility device can roll away out of control.

 Place mobility device's rear frame on a supporting wooden block before you remove wheels.

CAUTION!

- Risk of burning if power cable is shorted
 Turn off the power completely before removing any power supply components of the mobility device. Therefore take out batteries.
- Avoid bridging of contacts during measurements on live electrical components.



CAUTION! Any changes to the drive program can affect


- the driving characteristics and the tipping stability of the mobility device
- Changes to the drive program may only be carried out by trained Invacare[®] specialist providers.
- Invacare[®] can only give a warranty for safe mobility device driving behavior - especially tipping stability - for unaltered standard drive programs.

\triangle

CAUTION! Risk of accident

Accidental rolling can lead to accidents. – Secure mobility device against rolling away.

- The controls system is supplied with a standard drive program. If you have carried out customer-specific modifications to the drive program, you will have to make these changes again after installing the new power module.
- $\begin{tabular}{ll} \widehat{l} & When disassembling, note position of small parts such as screws and washers. Put small parts down so that they can be reassembled in right sequence. \end{tabular}$
- The plugs on the electronic module cannot be wrongly connected because all plugs have a different size and only fit in one socket.
- 10 mm wrench
 - 17 mm wrench
 - Rubber hammer
 - Phillips screwdriver
 - Supporting wooden block
 - To adapt the drive program: programming software or hand programming device and electronic system installation manual, available from Invacare.
- 1. Secure mobility device against rolling away.
- 2. Place supporting wooden block under frames at battery holder height.
- 3. Remove seat.
- Remove shroud.
 5.

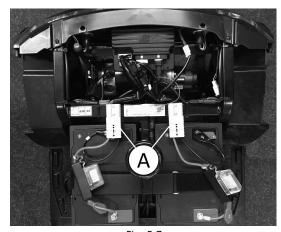



- Disconnect battery cable (1).
- 6. Loosen and remove rear wheel fixing screws.
- 7. Remove rear wheels.

- 8. Remove drive unit. See 5.11.1 Replacing drive unit, page 29.
- 9.

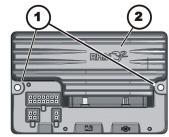


Fig. 5-8 DS90/DS120:

Remove screws (1) on power module (2).

10.





- Replace power module (2).
- 11. Install parts in reverse order.
- 12. Load drive program onto power module. See controls software description.
- 13. Test all functions (trial run).

5.7 Steering

5.7.1 Replacing steering column


CAU

CAUTION! Risk of accident

Accidental rolling can lead to accidents. – Secure mobility device against rolling away.

- When disassembling, note the position of small parts such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.
- 10 mm wrench
 - 17 mm wrench
 - 19 mm wrench
 - 32 mm wrench
 - Phillips screwdriver
 - Supporting wooden block
- 1. Remove seat.
- 2. Remove shroud.


Disconnect battery cable.

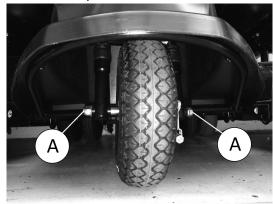

Place a supporting wooden block under the front part of the frame so that front wheel is no longer touching floor.


Replacing steering column

1.

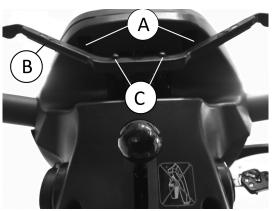

Remove drive lever screws ©.


- 2. Remove drive lever ^(B).
- 3. Loosen and remove screws (A) on operating console.
- 4. Remove operating console.
- 5.

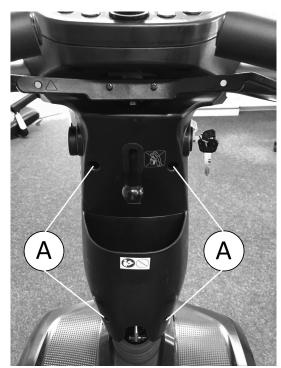

Remove screws (A) on rear steering column shroud.6. Remove steering column shroud.

7. Loosen screws on front basket holder.

- 8. Remove basket holder.
- 9. Remove spacer bushes.
- 10. Disconnect cable from wiring harness.
- 11.

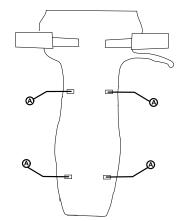

Remove screws (A) on front steering column shroud.

- 12. Remove the front steering column shroud.
- 13. Loosen brake cable. See 5.10.1 Replacing hand brakes, page 28.





Loosen and remove screw at clamp (1).


- 15. Unscrew fixing bolt for steering column and remove.
- 16. Pull steering column out upwards.
- 17. To replace fork 3–wheel, see 5.9.2 Replacing fork 3–wheel, page 25.

5.7.2 Replacing steering link/coupling rod

CAUTION!

Risk of accident Accidental rolling can lead to accidents.

- Secure mobility device against rolling away.
- When disassembling, note the position of small parts such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.
- 10 mm wrench
 - 12 mm wrench
 - 13 mm wrench
 - Phillips screwdriver
 - Circlip pliers
 - Supporting wooden block
- 1. Remove seat.
- 2. Remove shroud.

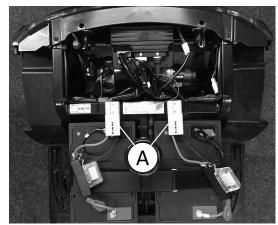


Disconnect battery cable A.

4. Remove steering column and chassis. See 5.7.1 *Replacing steering column, page 19.*

Replacing steering link/coupling rod

1.

3.

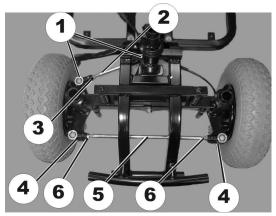


Remove circlip (1) on steering link (2).

- 2. Replace steering link.
- 3. Adjust length of steering link using setting screws (3).
- 4. Remove circlip (4) on coupling rod (5).
- 5. Replace coupling rod.
- 6. Install steering link and/or coupling rod in reverse order.
- 7. Test function (trial run).

Adjusting camber

- 1. Loosen screws (A).
- 2.

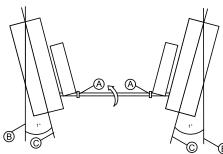

Spin steering link up by 0,5 cm. Wheels stand turned inwards by 1° \bigcirc from neutral position B.

- 3. Install parts of steering link and/or coupling rod in reverse order.
- 4. Test all functions (trial run).


5.7.3 **Replacing Steering Bar**




CAUTION! Risk of accident

Accidental rolling can lead to accidents. - Secure scooter against rolling away.

- When removing, take care of small parts such as ĩ screws and washers. Put all small parts down so that they can be installed in correct sequence.
- 32 mm wrench łĭ
 - Phillips screwdriver
 - Supporting wooden block
- 1. Remove seat.
- 2. Remove shroud.
- 3.

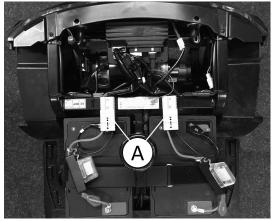

- Disconnect battery cable A.
- Dismantle steering link and coupling rod. See 4. Replacing steering link/coupling rod.
- 5.

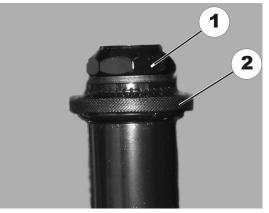

- Loosen screw (1).
- Loosen screw (2). 6.
- Replace steering bar. 7.
- 8. Install parts in reverse order.
 - Ĭ Tighten screw (1) to 25 Nm.
- 9. Apply brake, tighten wheel bolts at same time.
- 10. Adjust tracking. See Replacing steering link/coupling rod.
- 11. Test all functions (trial run).


5.7.4 Replacing curve control device



CAUTION! Risk of accident

Accidental rolling can lead to accidents. - Secure mobility device against rolling away.

- When disassembling, note the position of small ĭ parts such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.
- Phillips screwdriver Ĭ 32 mm wrench
- 1. Remove seat.
- 2. Remove shroud.

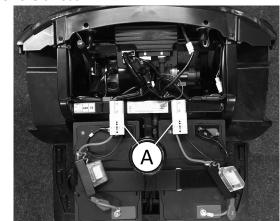

Disconnect battery cable A.

4.


3.



Remove front shroud colored cap A.


You can see curve control device © through opening 5. D.

- 6. Remove screw [®] that fixes shroud between front wheels.
- 7.

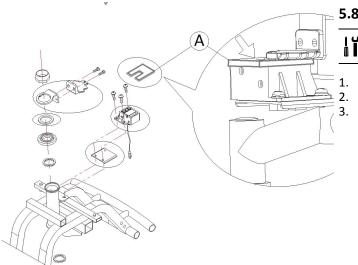

To check position of sensors of curve control device, use measuring gauge B supplied in "Bend speed reducer retrofitting kit".

Replacing sensors (5) without mounting

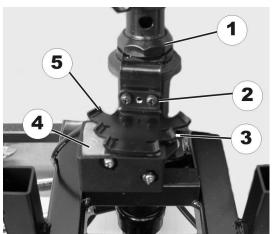

- 1. Loosen screws (2).
- 2. Replace sensor.

Replacing sensors (5) with mounting

- 1. Loosen fork bolt (1).
- 2. Replace sensors (5) with mounting.

Replacing regulating unit (4)

- 1. Loosen screws.
- 2. Remove sensor (5).
- 3. Loosen screws (3).
- 4. Replace regulating unit (4).

Adjusting

- 1. Set steering to straight ahead.
- 2. Align sensor straight using regulating unit.
- 3. Take hold of the sensor and tighten the fork screw.


Check

- 1. Turn steering completely to right and then left.
- 2. When steering is turned to its limit, the 3 regulating unit switches should be visible.
- 3. Install parts in reverse order.
- 4. Check functions (trial run).


5.8 Lighting unit

5.8.1 Replacing headlight

- Phillips screwdriver
- Remove seat.
- Remove shroud.

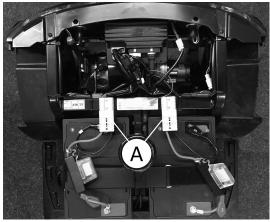

Disconnect battery cable A.

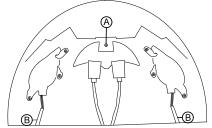


Remove screw (1) or $\ensuremath{\textcircled{}}$ on headlight under front shroud.

5. Remove headlight.

4.

Disconnect cable from wiring harness.
 T.

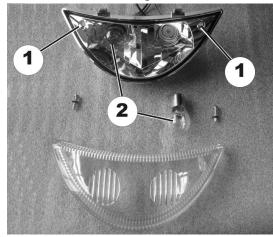


Remove screws (1) on headlight glass.

- 8. Remove headlight glass.
- 9. Replace bulb(s) in headlight.
- 10. Test function.

5.8.2 Replacing front direction indicators

Phillips screwdriver


- 1. Remove seat.
- 2. Remove shroud.
- 3.

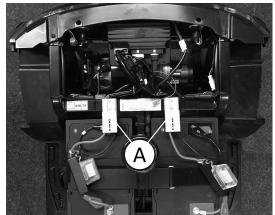

Disconnect battery cable A.

3-wheel version



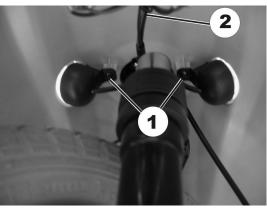


Remove direction indicator screw (1).

- 2. Disconnect direction indicator cable (2) from wiring harness.
- 3. Replace direction indicator.
- 4. Install direction indicators in reverse order.
- 5. Test function.


4-wheel version

- 1. Remove steering column shroud and steering column. See 5.7.1 Replacing steering column, page 19.
- 2. Remove chassis bolts and fold chassis upwards.
- 3.

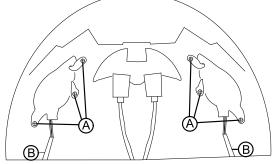

Remove direction indicator screw (A) on chassis.

4. Disconnect direction indicator cable [®] from wiring harness.

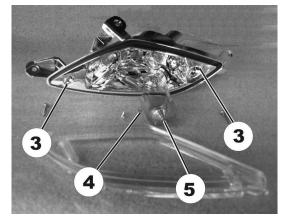

Remove screws (3) on glass.

6. Remove glass.

5.


- 7. Loosen screws (4) on orange-colored cap (5) and remove cap.
- 8. Replace lightbulb.
- 9. Install parts in reverse order.
- 10. Test function.

5.8.3 Replacing center brake light

CAUTION!

- Risk of burning if power cable is shorted
 Turn off the power completely before removing any power supply components of the mobility device. Therefore take out the batteries.
- Avoid bridging of contacts during measurements on live electrical components.
- When disassembling, note the position of small parts such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.
- Phillips screwdriver
- 1. Remove seat.
- 2. Remove shroud.


3.

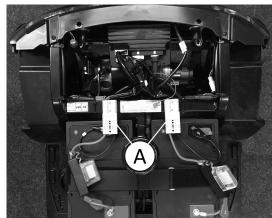


Disconnect battery cable A.

4.

5.

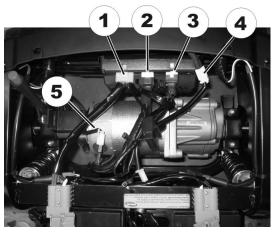

Disconnect plug (4).

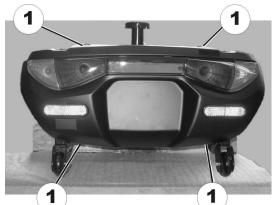

Remove bolts (1) on rear shroud.

- 6. Lift the rear shroud upwards.
- 7.



Remove screws (1) on brake light.

- 8. Replace brake light.
- 9. Install parts in reverse order.
- 10. Test functions (trial run).


5.8.4 Replacing rear lights/direction indicators

CAUTION!


- Risk of burning if power cable is shorted
 Turn off power completely before removing any power supply components of mobility device. Therefore take out batteries.
 - Avoid bridging of contacts during measurements on live electrical components.
- When disassembling, note the position of small parts such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.

• Phillips screwdriver

- 1. Remove seat.
- Remove shroud.
 Remove shroud.

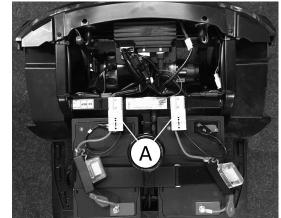

Disconnect battery cable A.

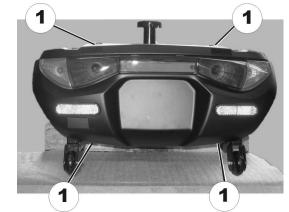


- Remove bolts (1) on rear shroud.
- 5. Lift rear shroud upwards.
- 6.

4.




Remove screws (1) on rear light.

Remove rear light.
 8.

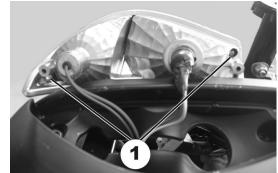

Remove screws (1) on glass.

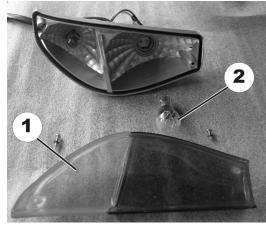

Remove glass (1).

- 10. Replace lightbulbs (2).
- 11. Install parts in reverse order.
- 12. Test function.

9.

5.9 Wheels

5.9.1 Replacing Front Wheel Suspension (4–Wheel Version)

CAUTION!

Risk of Uncontrolled Movement of Mobility Device

- Turn off power supply (ON/OFF key).
- Engage drive.
- Before raising mobility device, secure wheels by blocking them with wedges.
- Prevent the mobility device tipping by propping it up on a wooden block which is long and wide enough under the battery case. If the wooden block is too short or too high, the mobility device can still tip.

When removing, take care of small parts such as screws and washers. Put small parts down so that they can be installed in correct sequence.

- 10 mm wrench
 - 13 mm wrench
 - 17 mm wrench
 - 19 mm wrench
 - 32 mm wrench
 - Phillips screwdriver
 - Supporting wooden block

Tightening torques

Nut	Tightening torque	
(A), (C)	10 Nm	
8, D, E	15 Nm	



- 1. Remove seat. See corresponding chapter in user manual.
- 2. Remove front shroud. See Replacing front shroud.
- 3. Unplug battery cable.
- 4. Place a supporting wooden block under the front part of the frame so that the front wheel is no longer touching the floor.
- 5. Remove wheels. See *5.9.7 Removing Wheels, page 27*. 6.

Fig. 5-11


Loosen and remove nuts A.

- 7. Remove bolts and shock absorbers.
- 8. Loosen and remove nuts D and E that fix two steering links.
- 9. Loosen and remove nuts (B) on cross members on both sides.
- 10. Loosen and remove nuts $\ensuremath{\mathbb{C}}$ that fix cross members to frame.
- 11. Replace suspension.
- 12. Install parts in reverse order. Tighten screws and nuts to prescribed torque. See *Tightening torques, page 25*.
- 13. Test all functions.

5.9.2 Replacing fork 3–wheel

- 1. Prepare steering column. See 5.7.1 Replacing steering column, page 19.
- 2. Remove rubber mat in foot pan.
- 3. Loosen screws in chassis below rubber mat.
- 4. Remove chassis upwards.
- 5. Dismantle front wheel. See *5.9.3 Replacing front axle, page 26.*

6.

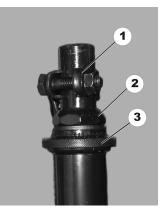

Pull off clamp (1).

- Loosen fork bolt (2). 7.
- Loosen knurled screw (3). 8.
- 9. Replace fork.
- 10. Install parts in reverse order.
- 11. Apply brake and tighten axle bolts at same time.
- 12. Test functions (trial run).

Replacing front axle 5.9.3



Risk of accident


CAUTION!

Accidental rolling can lead to accidents. - Secure mobility device against rolling away.

- When disassembling, note the position of small ĭ parts such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.
- The front axle can only be replaced on the 3-wheel ĭ version. The 4-wheel version does not have a replaceable axle.
- 17 mm wrench łĭ
- 19 mm wrench
 - Supporting wooden block
- 1. Remove seat.

5.

- 2. Remove shroud.
- 3. Disconnect battery cable.
- Place a supporting wooden block under the front part 4. of the frame so that the front wheel is no longer touching the floor.

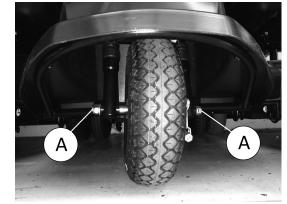

Loosen axle bolts (1) simultaneously on both sides.



Pull front axle slowly out.

- 7. Replace front axle.
- 8. Replace parts in reverse order.
- 9. Test function (trial run).

Removing Front Shock Absorber 5.9.4 (3–Wheel Orion^{PRO} only)



CAUTION! **Risk of accident**

When parking mobility device on antitippers, mobility device is no longer slowed by motor brake. Mobility device can roll away out of control.

- Place mobility device rear frame on a supporting wooden block before you remove wheels.




CAUTION! **Risk of accident**

Accidental rolling can lead to accidents.

- Secure mobility device against rolling away.
- When removing, take care of small parts such as ື່ງໃ screws and washers. Put all small parts down so that they can be installed in correct sequence.

łĭ 13 mm wrench

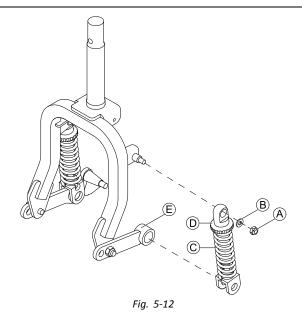

- Remove front axle. See 5.9.3 Replacing front axle, 1. page 26.
- 2. Remove screw (A) and washer (B).
- 3. Remove shock absorber \mathbb{C} from front fork \mathbb{E} .
- 4. Use setting screw D to set spring hardness.
- 5. Install parts in reverse order.
- Test all functions. 6.

5.9.5 **Replacing Rear Shock Absorber**

4.

5.

CAUTION!

Risk of accident

When parking mobility device on antitippers it is no longer slowed by motor brake. The mobility device can roll away out of control.

- Place mobility device rear frame on a supporting wooden block before you remove wheels.



CAUTION! **Risk of accident**

Accidental rolling can lead to accidents. - Secure the scooter against rolling away.

- When disassembling, note position of small parts ĭ such as screws and washers. Put small parts down so that they can be reassembled in right sequence.
- The drive motor needs to be removed before you Ĭ can replace the left-hand shock absorber. See 5.11.1 Replacing drive unit, page 29. If the right-hand shock absorber is to be replaced, you do not need to remove the drive motor.
- łĭ 5 mm Allen keys (2x)
- 1. Remove wheel. See 5.9.7 Removing Wheels, page 27.
- 2. Remove mud guard. See 5.11.1 Replacing drive unit, page 29.
- 3.


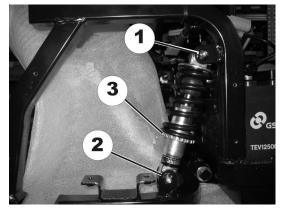


Fig. 5-13


- Loosen top bolt (1) on shock absorber.
- Loosen bottom bolt (2) on shock absorber. 4.
- 5. Replace shock absorber.
- 6. Use setting screw (3) to set spring hardness.
- 7. Install parts in reverse order.
- Test all functions. 8.

5.9.6 Replacing swing arm

- 14 mm wrench łĭ
 - 5 mm Allen key
 - Thread locking adhesive (for example Locktite 243)
- 1. Remove rear shroud.
- Remove drive unit. See chapter 5.11.1 Replacing drive 2. unit, page 29.
- 3. Remove batteries. See user manual.

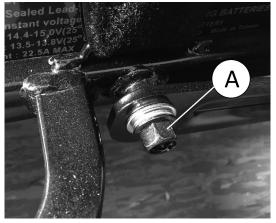

Loosen and remove screw A that attaches swing arm to chassis on left and right side of the mobility device.

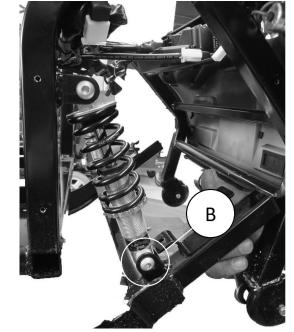

Loosen and remove lower bolt [®] on rear shock absorber on left and right side of the mobility device.

- Remove and replace swing arm. 6.
- 7. Install parts in reverse order.
- Apply thread locking adhesive to screw (A) and tighten 8. to 25 Nm.
- 9. Tighten screws.

5.9.7 **Removing Wheels**

- 19 mm wrench łĭ
 - 17 mm wrench
 - Rubber hammer
 - Supporting wooden block



Fig. 5-14

- Raise mobility device and place wooden blocks 1. underneath to support it.
- Remove wheel locknut (1). 2.
- 3. Remove wheel by tapping it gently with rubber hammer on the rear face to carefully loosen it from the axle.

CAUTION!

Risk of injury from wheels coming loose If wheels are insufficiently secured during mounting, they can come loose when driving.

- When mounting wheels, tighten bolts with 30 Nm.
- Never use normal nuts instead of self-locking nuts.
- Always use new bolts with an undamaged coating.

Problems when removing wheel? ື່ງໃ

- It may be necessary to use a special tool. Ask your Invacare provider to help you.

5.9.8 **Repairing Tyre Punctures**

- Inner tube repair set or a new inner tube łĭ
 - Talcum powder
 - 12 mm wrench (depending on rim type)
 - 10 mm wrench (depending on rim type)
- Remove wheel. See 5.9.7 Removing Wheels, page 27. 1. 2. Remove valve cap.
- 3.
- Deflate tyre by pressing in the center valve pin.

CAUTION! **Risk of explosion**

The wheel explodes if the air pressure has not been released from the wheel before the wheel rim is removed.

- Always let all the air out of the tyre before removing the rim.

- Loosen and remove screws. 4.
- Remove both wheel rim halves out of tyre and remove 5. inner tube.
- Repair inner tube and refit in wheel, or replace it with 6. new inner tube.
 - If the old inner tube has been repaired and is to Ĭ be used again, and became wet during repair, it is easier to replace it if it is lightly dusted with talcum powder beforehand.
- 7. Refit wheel rim parts from outside into tyre.
- 8. Pump up tyre lightly.
- Reinsert nuts and bolts which hold the wheel rim 9. together and tighten fully.
- 10. Make sure that tyre is properly located on wheel rim.
- 11. Inflate tyre up to recommended tyre pressure.
- 12. Check to make sure that tyre is still located properly on wheel rim.
- 13. Screw valve cap back on.
- 14. Locate wheel on drive shaft again.

- 15. Apply handbrake and hold it there to center brake.
- 16. Tighten wheel locknut with handbrake applied.



- Risk of injury from wheels coming loose If wheels are insufficiently secured during mounting, they can come loose when driving.
- When mounting wheels, tighten bolts with 30 Nm.
- Never use normal nuts instead of self-locking nuts.
- Always use new bolts with an undamaged coating.

5.10 Brakes

5.10.1 Replacing hand brakes



CAUTION! **Risk of accident**

Accidental rolling can lead to accidents. - Secure the scooter against rolling away.

- When disassembling, note the position of small ĵ parts such as screws and washers. Put small parts down so that they can be reassembled in the right
- The front axle can only be replaced on the 3-wheel ĭ version. The 4-wheel version does not have a replaceable axle.
- 10 mm wrench łĭ


sequence.

- 17 mm wrench
- 19 mm wrench
- Supporting wooden block
- 1. Remove seat.
- Remove shroud. 2.
- 3. Disconnect battery cable.
- Place a supporting wooden block under the front part 4. of the frame so that the front wheel is no longer touching the floor.

3-wheel version

- Slacken brake cable. See 5.10.2 Replacing brake cable, 1. page 29.
- Disconnect brake cable at wheel. 2.



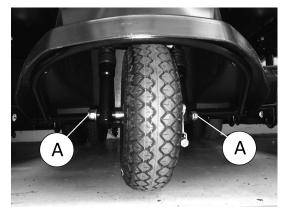

- Loosen axle bolts (A) simultaneously on both sides.
- 4. Pull front axle slowly out.
- 5. Replace brake lining or complete brake.

- 6. Reassemble wheel and brake.
- 7. Push axle through.
- 8. Apply brake and tighten axle bolts at same time.
- 9. Install parts in reverse order.
- 10. Adjust braking force on wheel using brake lever setting screw.
- 11. Test braking function while pushing.

4-wheel version

- 1. Slacken both brake cables.
- 2. Disconnect brake cable at affected wheel.
- 3. Loosen wheel bolt.
- 4. Pull wheel and brake off shaft.
- 5.



Pull brake out of brake drum.

- 6. Replace brake lining or complete brake.
- 7. Reassemble wheel and brake.
- 8. Push wheel and brake onto shaft.
- 9. Apply brake and tighten axle bolts at same time.
- 10. Install parts in reverse order.
- 11. Adjust braking force on wheel using brake lever setting screw.
- 12. Test braking function while pushing.

5.10.2 Replacing brake cable




CAUTION! Risk of accident

Accidental rolling can lead to accidents. – Secure mobility device against rolling away.

- When disassembling, note position of small parts such as screws and washers. Put small parts down so that they can be reassembled in right sequence.
- $\underbrace{\overset{\circ}{\mathbb{I}}}_{} \qquad \mbox{You do not need to remove wheel to replace brake cable.}$

ĬĬ	٠	10 mm wrench
	٠	Phillips screwdriver

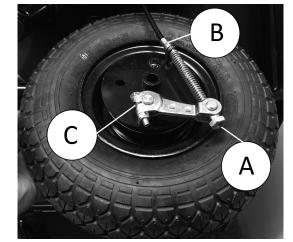

Slacken brake cable using setting screw (A).

- Disconnect brake cable at wheel ^B.
- 3. Disconnect brake cable at brake lever on steering column.
- 4. Route brake cable through shroud.
- 5. Loosen brake lever screw C at wheel.
- 6. Replace brake cable.

1.

2.

- 7. Install parts in reverse order.
- 8. Test braking function while pushing.

5.11 Drive Components

5.11.1 Replacing drive unit

CAUTION!



Risk of accident When parking mobility device on antitippers it is no longer slowed by motor brake. Mobility device can roll away out of control.

 Place mobility device rear frame on a supporting wooden block before you remove wheels.




CAUTION!

Risk of accident Accidental rolling can lead to accidents. – Secure mobility device against rolling away.

- When disassembling, note the position of small parts such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.
- The plugs on the power module cannot be wrongly connected because all plugs have a different size and only fit in one socket.
- 10 mm wrench
 - 17 mm wrench
 - 17 mm socket wrench
 - 10 mm socket wrench
 - Oblique pliers
 - Rubber hammer
 - Phillips screwdriver
 - Supporting wooden block
 - 5 mm Allen key
 - Thread locking adhesive (for example Locktite 243)

- 1. Secure mobility device against rolling away.
- 2. Place supporting wooden block under the frame at battery holder height.
- 3. Remove seat.
- 4. Remove shroud.
- 5.

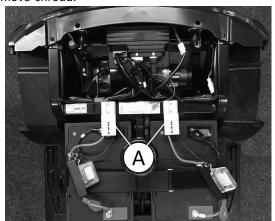


Disconnect battery cable A.

- 6. Unscrew rear wheel fixing bolts.
- 7. Remove rear wheels.
- 8. Cut cable ties.


9.

10.

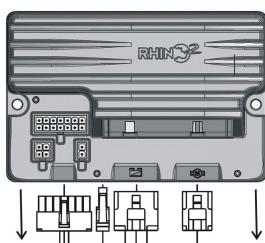


Disconnect plugs from power module.

- Disconnect motor plug.
- Disconnect plug for electromagnetic brake.
- Disconnect speedometer plug.

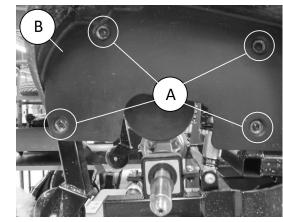


- Remove screws (A) on mud guard.
- 11. Remove mud guard B.

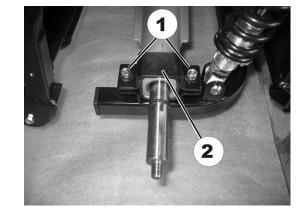

Remove screws (1) on drive holder (2).

Remove drive holder (2).
 14.


12.

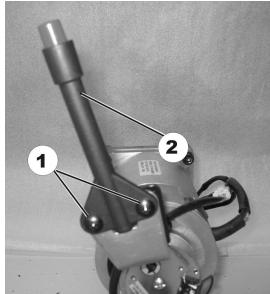

- Remove screws (1) on disengaging lever (2).
- 15. Remove disengaging lever (2).
- 16. Place supporting wooden block under swing arm.

Replacing drive motor

1.

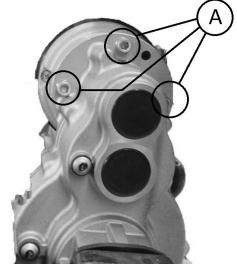


Loosen three Allen screws A.

- 2. Lift motor out of holder and take it out of the frames to the side.
- 3. Replace motor.

Replacing drive unit

1.

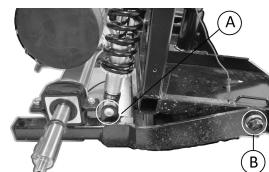


Loosen and remove screw (B) on swing arm. For replacing swing arm, see 5.9.6 Replacing swing arm, page 27.

- 2. Loosen and remove lower bolt (A) on spring.
- 3. Lift drive unit out of holder and take it out of the frames to side.
- 4. Replace drive unit.

Installation

1. Install parts in reverse order.

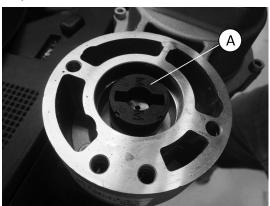

CAUTION!

Reduced function when clutch is mounted incorrectly





CAUTION!

- 2. Apply thread locking adhesive (for example Locktite 243) to screw ^(B) on swing arm and tighten to 25 Nm.
- 3. Test all functions (trial run).


5.11.2 Replacing carbon brushes

 $\overset{\circ}{l} \qquad \text{Always replace all the carbon brushes on both} \\ \text{motors.}$

- 5 mm Allen key
 - 6 mm Allen key
 - 10 mm socket wrench19 mm socket wrench
 - Blade screwdriver

Long wooden blocks, min. 14 x 14 x 30 cm

Removing carbon brushes

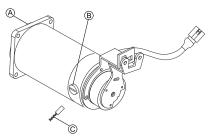

- 1. Switch off mobility device.
- 2. Remove drive wheels.
- 3. Disengage motor (A) by setting the engaging lever to "Push".
- 4. Loosen and remove all four plastic caps ^(B).
 - $\underbrace{\overset{\circ}{\fbox}}_{\text{carbon brushes.}} \text{ Note the fixing position and location of the carbon brushes.}$
 - Used carbon brushes need to the refitted exactly in the same position from which they were taken in order to guarantee optimum contact to the collector.
- 5. Make a marking on the motor and the carbon brushes in order to guarantee correct installation.
- 6. Remove carbon brushes completely from mounting ©.
- 7. Check carbon brush and spring for level of wear, broken components or discolouration.

Installing carbon brushes

- Depending on condition of brush and spring: either install brushes in exactly the same position from which they were taken or - fit new brushes.
- 2. Install plastic caps and tighten them firmly.
- 3. Install drive wheels..
 - The following procedure is necessary to run carbon brushes in after replacement and thus guarantee maximum performance.

CAUTION!

Risk of accidents Injuries to workers, damage to surroundings and mobility device.

- Do not leave mobility device unattended during the following procedure.
- Secure area.
- 4. Lift mobility device up on one side and get a second person to place a wooden block around 14 cm high underneath it so that the drive wheel is suspended freely.
- 5. Repeat on other side of mobility device.
- 6. Allow motors to run for an hour in forward direction.
- 7. Allow motors to cool down for 30 minutes.
- 8. Allow motors to run for an hour in reverse direction.
- 9. Lift mobility device off wooden blocks.

5.12 Seat system

5.12.1 Replacing seat suspension/spring



CAUTION! **Risk of accident**

Accidental rolling can lead to accidents. - Secure scooter against rolling away.

- When disassembling, note position of small parts ĵ such as screws and washers. Put small parts down so that they can be reassembled in the right sequence.
- The following seat support tubes are available for ĵ the seat suspension:
 - 508.5 mm ٠
 - 533.5 mm
 - 558.5 mm •




Risk of tipping

Orion^{PRO} 3–wheel could tip over when driving uphill when incorrect seat support tube/seat suspension is installed.


- Always install seat support tube with two holes only.
- Always install seat support tube medium or short when retrofitting.

ł۲ 17 mm wrench (2x) •

- Secure mobility device against rolling away. 1.
- 2. Remove seat.
- 3.

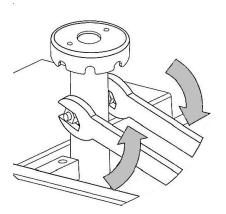


Loosen securing bolt of seat support tube.

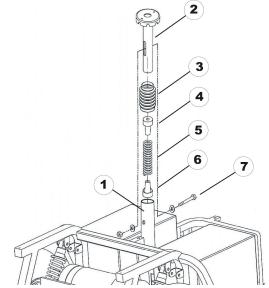

Remove bolt (7), nut and washers.

- Remove seat support tube (2) from receptacle tube (1). 5.
- Remove bellows (3) including spring (5) and plastic 6. end pieces (4 and 6).
- 7. Fit plastic end pieces (4 and 6) to new spring.
- 8. Grease spring generously. Spring should be covered completely.
- 9. Assemble bellows (3).
- 10. Install bellows (3) including spring (5) and plastic end pieces (4 and 6).
- 11. Insert seat support tube (2) into receptacle tube (1) and press down gently.
- 12. Reposition bolt (7), nut and washers.
- 13.

4.

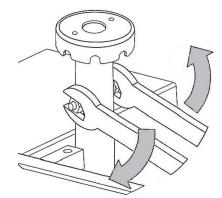

Firmly tighten securing bolt of seat support tube. 14. Reposition seat.

5.12.2 Replacing seat support tube

łĭ 17 mm wrench (2x)

CAUTION! ∕!∖

Risk of tipping

Orion^{PRO} 3–wheel could tip over when driving uphill when incorrect seat support tube/ seat suspension is installed.

- Always install seat support tube with two holes only.
- Always install seat support tube medium or short when retrofitting.
- 1. Remove seat.
- 2. Remove shroud. See "Removing shroud".



Remove securing bolt, nut and washers of seat support tube.

- Replace seat support tube. 4.
- Adjust seat height. 5.
- Reinsert securing bolt, nut and washers and tighten. 6.

5.12.3 Adjusting Seat System to Different **Payloads**

This mobility device covers a wide range of different payloads.

Corresponding to the payload, different components need to be replaced or adjusted. Check the following table for the different use ranges:

rayioad Ranges of components			
Component	Payload range 1	Payload range 2	
Seat suspension	≤ 100 kg	≤ 160 kg	
Sliding rail (Orion ^{PRO} 3–wheel only)	≤ 127 kg (150 mm sliding rail length)	≤ 150 kg (60 mm sliding rail length)	
Installation position of seat	≤ 150 kg (rear hole of rear hole pattern on seat)	> 150 kg (middle hole of rear hole pattern on seat)	

Payload Ranges of Components



CAUTION! Risk Of Tipping

Orion^{PRO} 3-wheel could tip over when driving uphill when incorrect seat support tube/ seat suspension is installed.

- Always install seat support tube with two holes only.
- Always install seat support tube medium or short when retrofitting.

ĬĬ	٠	TX30 Torx key
	•	12 mm wronch

- 13 mm wrench
 - 17 mm wrench (2x)

Removing Seat



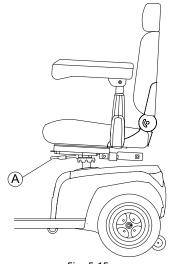


Fig. 5-15

Pull lever A to disengage seat.

- Hold the seat firmly by the backrest and front edge 2. and remove it upwards.
- 3. Remove things that are in the way, such as armrests.
- 4. Secure seat in a way that you can reach lower side.
- 5.

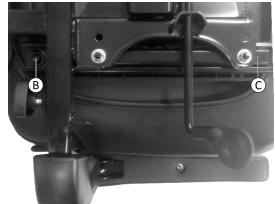


Fig. 5-16



Remove screws (B) and (C) on both sides. If required, move seat bracket forwards and backwards to reach screws.


Remove seat bracket including sliding rails from seat. 6.

Replacing Seat Suspension / Spring

The mobility device is fitted either with a fixed seat support tube or with a seat suspension. Only the seat suspension needs to be replaced. The seat support tube fits to all payloads.

1. Replace seat suspension with correct one. See 5.12.1 Replacing seat suspension/spring, page 32.

Replacing Sliding Rails

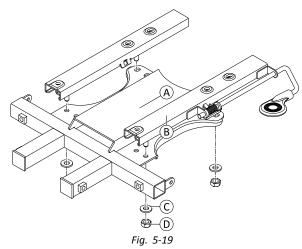

- 1. Loosen and remove nuts $\mathbb D$ including washers $\mathbb C.$
- 2. Remove sliding rails [®] from seat bracket [®].
- 3. Install new parts in reverse order.
- 4. Check all functions.

Installing Seat

The lower side of the seat is fitted with six screw holes within two hole pattern on each side. When installing, ensure to select the correct screw hole corresponding to the payload.



- Fig. 5-20 Rear hole pattern *(A*) and front hole pattern *(B*) on left-hand side of a seat. Installed seat might differ.
- 1. Install screws to corresponding screw hole of rear hole pattern (see *Payload Ranges of Components, page 33*).
- 2. Install remaining screws to front hole pattern.
- 3. Check all functions.

Notes

Invacare Distributors

Australia:

Invacare Australia Pty. Ltd. 1 Lenton Place, North Rocks NSW 2151 Australia Phone: 1800 460 460 Fax: 1800 814 367 orders@invacare.com.au www.invacare.com.au

United Kingdom:

Invacare Limited Pencoed Technology Park, Pencoed Bridgend CF35 5AQ Tel: (44) (0) 1656 776 222 Fax: (44) (0) 1656 776 220 uk@invacare.com www.invacare.co.uk Canada: Invacare Canada L.P. 570 Matheson Blvd East, Unit 8 CDN Mississauga, On. L4Z 4G4 Phone: (905) 890 8300 Toll Free: 800.668.5324 www.invacare.ca

Eastern Europe, Middle East & CIS:

Invacare EU Export Am Achener Hof 8 D-88316 Isny Tel: (49) (0)7562 700 397 eu-export@invacare.com www.invacare-eu-export.com Ireland: Invacare Ireland Ltd, Unit 5 Seatown Business Campus Seatown Road, Swords, County Dublin Tel : (353) 1 810 7084 Fax: (353) 1 810 7085 ireland@invacare.com www.invacare.ie

New Zealand:

Invacare New Zealand Ltd 4 Westfield Place, Mt Wellington 1060 New Zealand Phone: 0800 468 222 Fax: 0800 807 788 sales@invacare.co.nz www.invacare.co.nz



1604202-C 2020-10-01



Yes, you can.