
Invacare® Accessories

Invacare® Statio (L871)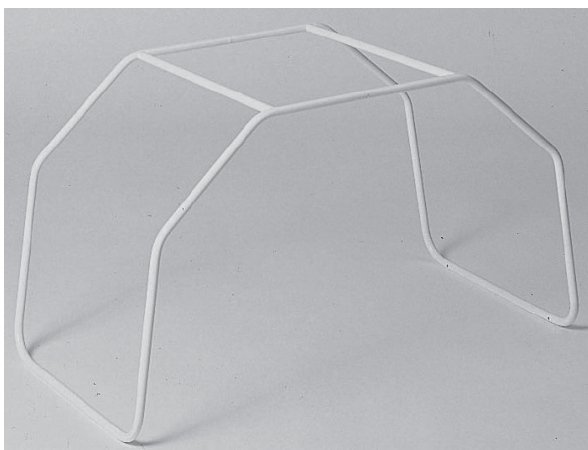

en	Bed Cradle User Manual	1
de	Bettbügel Gebrauchsanweisung	1
es	Armazón para cama Manual del usuario	2
fr	Arceau de lit Manuel d'utilisation	2
it	Archetto del letto Manuale d'uso	2
nl	Dekenboog Gebruiksaanwijzing	3
pt	Armação para cama Manual de utilização	3
bg	Рамка за легло РЪКОВОДСТВО	3

1 Safety

en

CAUTION!

- Check that the product has been delivered in perfect condition before you use it.
- If you are unsure about usage, adjustment or maintenance of the product, contact your Invacare distributor.
- Do not exceed the maximum load, shown on the product label.
- Do not use or store the product outdoors.
- Periodically check stability and rigidity of the product and, where applicable, that the screws are tightened.

Cleaning and disinfection

- Remove any dirt or dust using a soft, damp cloth.
- Use soap and water to remove stubborn stains.
- Dry with a soft cloth.
- For disinfection in hospital environments, follow the hygiene specialist's recommendations.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Usage

The bed cradle can be placed on the bed to alleviate pressure from the duvet on the legs or feet.

1. Position the cradle at the foot end above the patient.
2. Arrange the duvet over the cradle.

Technical data

Safe working load	25 kg
Frame material	Steel tube

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Invacare AB
Invacare Limited